

## Hello

Congratulation! You are about to experience an excellence product from agegracefullyshop.

## Warranty

One year limited warranty against manufacturing defects.

Please keep the receipt as proof of purchase.

## Contact Us

For service, spare parts and operational instructional video please visit our website or contact us:

Bekind Solutions Pte Ltd. Singapore

Call : +65 8876 8501

Email: [hello@agegracefully.shop](mailto:hello@agegracefully.shop)

Website: [www.agegracefully.shop](http://www.agegracefully.shop)



## Get Started


This walking stick comes complete with USB charging cable. We recommend that you charge the built-in battery before using it.

## Charging

Connect the USB charging cable to the Charging Port and Power source.

Charing indicator show Red light on while charging.

Charing indicator light off when charging is completed.

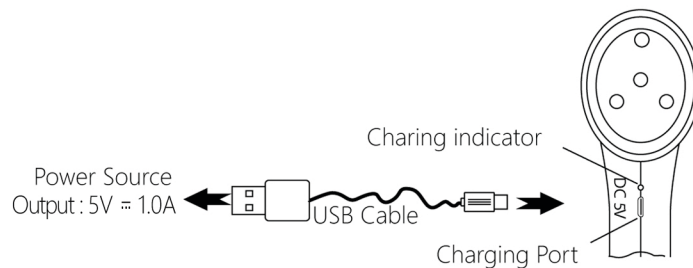

**Cautions:** Do not charge battery in damp or wet environment.

## Safety

This walking stick contains sensitive electronic components and can be damaged if burned, punctured, or crushed. Do not use a damaged walking stick, such as one with a cracks, visible water intrusion in the handle, as it may cause injury. Avoid heavy exposure to dust or sand.

This walking stick is designed to be used by adults. Children should not be allowed to tamper with the control or play with the product.

It is dangerous to alter the specifications or modify this product in any way. This product should be serviced by an authorised AgeGracefullyShop agent and only genuine spare parts should be used.

## Battery

When the built-in battery is fully charged:

The LED Head Light can last up to 10 hrs.

The LED Head Light & Red Flashing Light can last up to 5 hrs.

The Alarm can last up to 5 hrs.

The MP3 Player / Radio can last up to 5 hrs.

## Thoughtful Features

The High power LED Head Light (20 Lux) is enough to brighten up the walkway to improve visibility



The Red Flashing Light and the LED Head Light improve visibility for the oncoming vehicle on the user



Alarm attract quick attention of bypasses in time of emergency


**Essential Handle:** Alarm can be activated by the sliding switch on the top.

**MP3 Handle:** Upon activation, the Auto-Fall-Alarm activates automatically when the walking stick is in horizontal position.

## Essential Handle

Sliding Switches:


1. LED Head Light
2. Safety LED Head Light & Red Flashing Light
3. OFF
4. Alarm

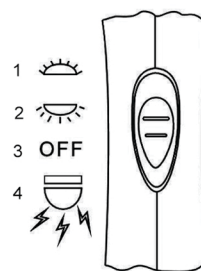

## MP3 Handle

Red Button:

1. LED Head Light
2. Safety LED Head Light & Red Flashing Light
3. OFF

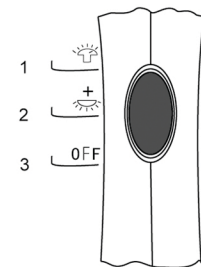

## MP3 Handle

(with Digital FM Radio and Auto-Fall-Alarm)

**Auto-Fall-Alarm activation:**


Press and hold the red button for about 3 sec (Beep Sound) to activate the auto fall alarm.

Alarm will be triggered when the walking stick is in horizontal position.

Press the red button to stop the Alarm.

Press and hold the red button again for about 3 sec (Beep Sound) to de-activate the auto fall alarm.

**Radio / MP3 Player:**

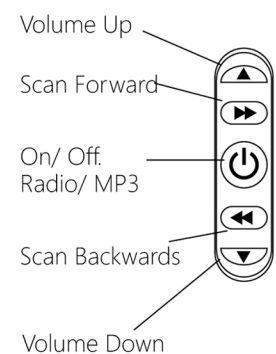


Press to switch on/ off the Radio/ MP3 functions.

Press and hold to switch between Radio and MP3 mode.


## LED Light Angle Adjustment

There are 4 different pivoting angles available for adjustment.

**Cautions:** Pinch point.

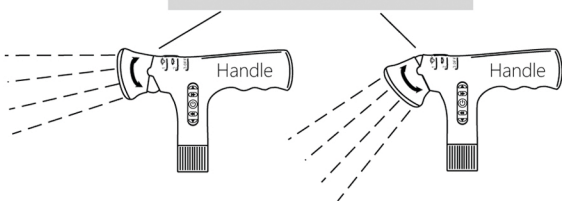


## Holding Handle The Right Way

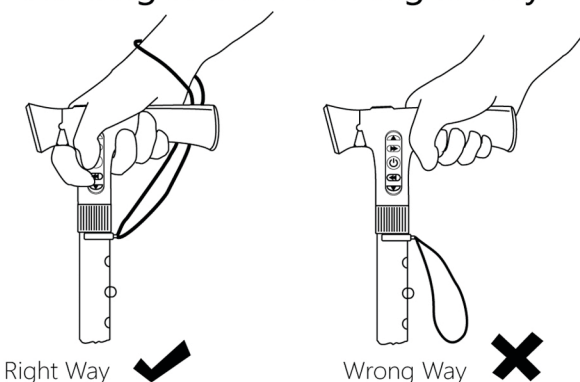


## Height Adjustment

1. Loosen the locking collar.
2. Press the Ball-Spring-Catch. Adjust the tubing to desire height with Ball-Spring-Catch.
3. Ensure the Ball-Spring-Catch is fix into position, next tighten the locking collar

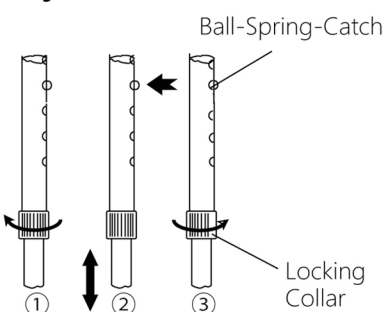


Height Adjustment Reference (cm)

	146	150	154	158	162	166	170	174	178	182	186
	75	77	79	81	83	85	87	89	91	93	95

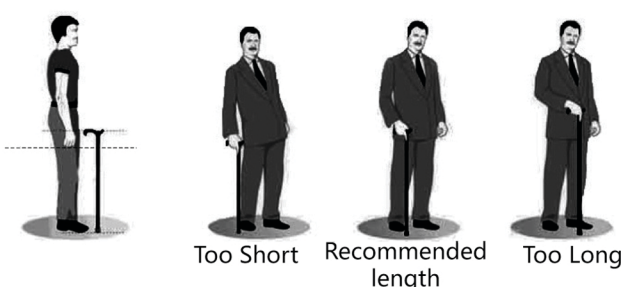

## SD Card

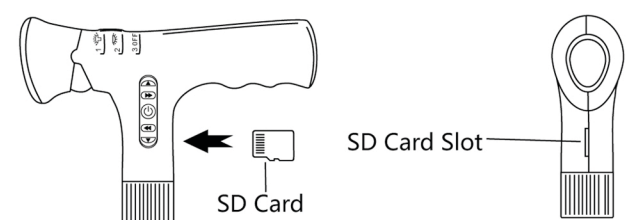

The SD Card is located in the SD Card Slot.

Press the SD Card to unlatch it before removal.

Only MP3 format is recognised in the MP3 Player.

**CE RoHS** BS 5205:1990

agegracefully ... Keep Moving ...