



AirFit™ F20



## Easily fits them and everyone in between

Finding the right mask fit for a broad range of patients can be challenging. That's why we created the AirFit™ 20 series – masks that are specially-designed to fit all patients, regardless of facial profile. The AirFit F20 is designed to perform across a wide range of therapy pressures, and fit a remarkable 97% of patients in a recent international fitting study.<sup>1-3</sup>



[ResMed.com/AirFitF20](https://www.ResMed.com/AirFitF20)



## AirFit F20 highlights

**Plush headgear**  
is specifically designed for extra softness and stability.

**InfinitySeal™ cushion**  
provides an effective seal across a wide range of therapy pressures.

**Magnetic clips**  
guide the headgear to the frame quickly and easily.

**Soft and flexible frame**  
sits underneath the eyes and adapts to different facial types, while its integrated padding provides added softness.

**Diffused venting**  
evenly disperses air to dampen air flow and noise out of the mask.

**Quick-release elbow**  
makes it simple to disconnect from the tubing without removing the mask.





AirFit F20 for Her  
Designed specifically for women.



ResMed is the #1 mask brand preferred by patients.<sup>4</sup>

## AirFit F20 components

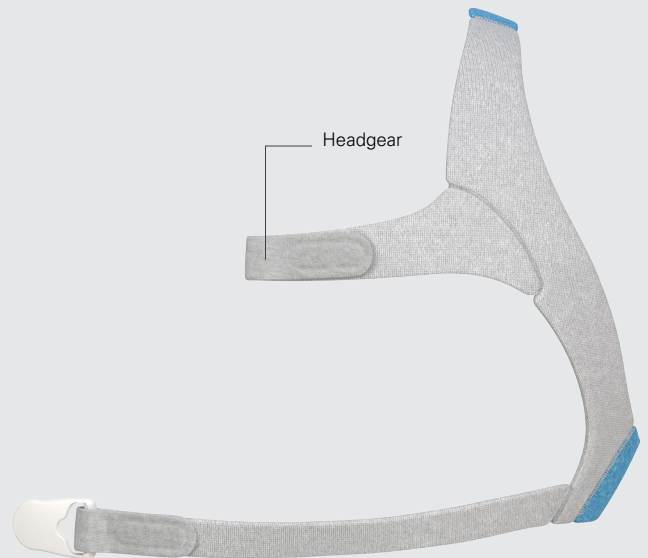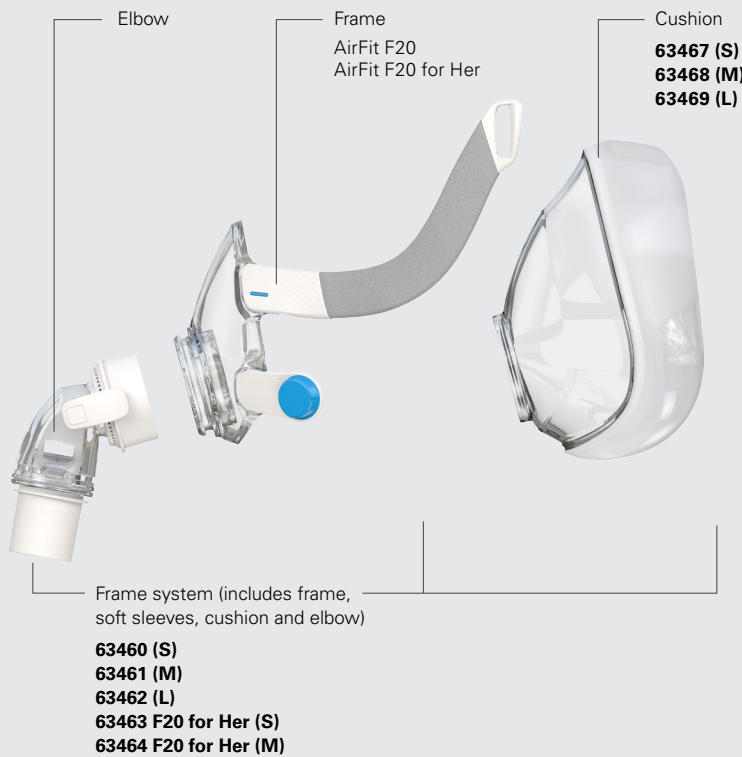

AirFit F20 headgear with headgear clips  
**63470 (S)**  
**63471 (Std)**  
**63472 (L)**



AirFit F20 for Her headgear with headgear clips  
**63473 (S)**

## Product codes

**Complete mask systems** (Includes frame system + headgear)

AirFit F20: Small	<b>63400</b>
AirFit F20: Medium	<b>63401</b>
AirFit F20: Large	<b>63402</b>
AirFit F20 for Her: Small	<b>63403</b>
AirFit F20 for Her: Medium	<b>63404</b>



## Fitting guide for AirFit F20



- Start by ensuring the magnets on the lower straps are unclipped.
- If they are clipped, detach them by twisting and pulling them away from the frame.



- Hold the mask to the face, ensuring the top of the mask is aligned with the top of the nasal bridge.
- Pull the headgear over the head, making sure the logo is facing outward.



- Position the lower straps so they both sit below the ears.
- Bring each of the magnets up to meet their corresponding clips on the frame.



- Adjust the fastening tabs on the upper straps until the mask feels even and stable.
- Reattach the fastening tabs.



- Repeat step 4 with the lower straps.
- Reattach the fastening tabs.



- Connect the device's air tube to the elbow, then click the elbow into the front of the mask.
- Start the device.



### Additional options

For patients with longer hair, we recommend feeding their hair through the back of the headgear. If tied in a pony tail, it should sit high enough to fit comfortably through the back of the headgear.

Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

1 ResMed AirFit F20 internal Australian fitting study of existing CPAP patients conducted between March–April 2016. 2 ResMed AirFit F20 internal USA fitting study of existing CPAP patients conducted April 2016. 3 ResMed AirFit F20 internal EU and APAC fitting study of existing CPAP patients conducted June 2016. 4 Combined responses of those expressing a preference in an independent patient survey in the UK, USA, France, Germany and Australia between April 10 to May 11, 2015. Visit [ResMed.com/MaskBrand](http://ResMed.com/MaskBrand).

Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See [ResMed.com](http://ResMed.com) for other ResMed locations worldwide. AirFit and InfinitySeal are trademarks and/or registered trademarks of the ResMed family of companies. Specifications may change without notice. © 2016 ResMed Ltd. 1019453/2 2016-10