

## Sizing Instructions Recommended by Sox For Horses!

- 1) Measure the feet that the Sox will be going on (Fronts or Hinds). Please keep in mind, if you are buying a set for both the Fronts and Hinds, the Hind feet are typically smaller than the front. In that case a mixed set may be required for best fit if there is a dramatic difference between the 2.



- Using a flat ruler or a tape measure, pick up the largest hoof. Looking at the flat, bottom of the hoof, measure from the front **tip of the toe to the end of the heel bulbs**. It is a straight line from the front to the rear of the hoof. Include the bulbs because they must also go through the socks.
- Then turn your ruler widthwise and measure the **widest width of the hoof**.
- Those are the two measurements you need to determine the size of the socks to order.

### To Note:

- Some horses can fit in-between sizes. In that case, socks can be custom fit.
- If the hoof measures too close to the top end of one size... but the next size up is too large, order the larger size up.
- Warmblood/Thoroughbred Sox cannot fit over a hoof that measures more than 7.00" Wide x 7.00" Long with a shoe. If the horse does not wear shoes, the socks may fit a 7.25" Wide x 7.25" Long foot. If your horse's hooves are larger than 7.00" x 7.00" with a shoe, please contact Sox For Horses! directly to discuss having custom Sox made.
- Quarter/Standard Sox are most likely to fit medium size horses. To be safe, measure the hooves. Larger Quarter Horses that may require a larger size, and Thoroughbreds with small hooves/delicate legs may be able to fit this size.

2) Once you have your horses' foot measurements, look at the chart to see which size most closely correlates:

- For Standard Donkey's, Mammoth Donkey's, and Mules please use the size chart below.
- For Miniature Horses/Donkeys please contact Sox For Horses! directly @ 1-850-907-5724.
- **If there is no mention of shoes by the hoof measurements, the socks will fit with shoes on or off.**

| WIDTH'' | LENGTH '' | With or Without Shoes | SOX SIZE               | Special Order |
|---------|-----------|-----------------------|------------------------|---------------|
| 7.25    | 7.25      | Without               | Warmblood/Thoroughbred |               |
| 7.25    | 7.25      | With                  | Special Order          | YES           |
| 7.00    | 7.00      | With                  | Warmblood/Thoroughbred |               |
| 6.75    | 6.75      |                       | Warmblood/Thoroughbred |               |
| 6.50    | 6.50      |                       | Warmblood/Thoroughbred |               |
| 6.25    | 6.25      |                       | Warmblood/Thoroughbred |               |
| 6.00    | 6.00      | With                  | Warmblood/Thoroughbred |               |
| 5.75    | 6.00      | With                  | Warmblood/Thoroughbred |               |
| 6.00    | 6.00      | Without               | Quarter/Standard       |               |
| 5.75    | 6.00      | Without               | Quarter/Standard       |               |
| 5.50    | 6.00      |                       | Quarter/Standard       |               |
| 5.25    | 6.00      |                       | Quarter/Standard       |               |
| 5.00    | 6.00      |                       | Quarter/Standard       |               |
| 5.75    | 5.75      |                       | Quarter/Standard       |               |
| 5.50    | 5.50      |                       | Quarter/Standard       |               |
| 5.25    | 5.25      |                       | Quarter/Standard       |               |
| 5.00    | 5.00      |                       | Quarter/Standard       |               |
| 4.75    | 5.00      | With                  | Quarter/Standard       |               |
| 4.75    | 5.00      | Without               | Quarter/Standard       |               |
| 4.75    | 4.75      |                       | Sport Pony             | YES           |
| 4.50    | 4.50      |                       | Sport Pony             | YES           |
| 4.25    | 4.25      |                       | Sport Pony             | YES           |
| 4.00    | 4.00      |                       | Sport Pony             | YES           |
| 3.50    | 4.00      |                       | Sport Pony             | YES           |

### **How-To Tell If The Socks Fit Correctly:**

Once you have the first sock on your horses' leg, place your hand between the sock and the cannon bone. It should not be constrictively tight.

If you use a plastic bag over the hoof to slide the sock up onto the leg, they should go on without a struggle. The first few times may be a bit a bit of a challenge but should still be manageable if your horse stands still.

The Sox For Horses! team is very knowledgeable and happy to answer detailed questions directly @

1-850-907-5724.