

IT'S ALL IN OUR NAME!

# MadaCide-FD

FAST DRYING (FD) and more...



▲ **One Step Infection Control Spray-Air Dry.**

▲ A Safe and Effective Alcohol Based Broad Spectrum Formula.

▲ 6 Minute TB Kill Time.

▲ 30 Second HIV Kill Time.

▲ 3 Minute Hepatitis B Kill Time.

**MadaCide-FD** HOSPITAL DISINFECTANT/CLEANER MEETING YOUR INFECTION CONTROL NEEDS FROM MADA. A NAME YOU HAVE TRUSTED FOR OVER 30 YEARS.

# **MadaCide-FD**

## **(FAST DRYING)**

*Another weapon in the fight for Infection Control.*

**MadaCide-FD**  
*is the newest member of the  
 Mada Family of Infection Control Products . . . and by the way . . .*  
**YOU ASKED FOR IT!**

### **HERE'S HOW FAST IT WORKS**

<b>ORGANISMS</b>	<b>MINIMUM CONTACT TIME</b>
Herpes Simplex II, HIV-I (Aids Virus) and Influenza Strain A2/HK	30 seconds
Hepatitis B Virus (DHBV), Polio I Virus, Pseudomonas aeruginosa, Rhinovirus, Salmonella choleraesuis and Staphylococcus aureus	3 minutes
Aspergillus niger, Candida albicans, Vancomycin-resistant Enterococcus faecalis (VRE), Canine Parvovirus and Trichophyton mentagrophytes	5 minutes
Mycobacterium bovis BCG (Tuberculosis)	6 minutes
Klebsiella pneumoniae	10 minutes

**MadaCide-FD** is EPA registered, assuring that all performance claims stated on the label are accurate and verifiable.

*\*It is Fungicidal, Pseudomonacidal, Virucidal, Bactericidal and Tuberculocidal.*

*\*(Complete Activity Spectrum Available)*

### **For Surgical Instrument Presoak:**

*Remove gross debris. Presoak surgical instruments in undiluted solution for 10 minutes. Proceed with normal decontamination procedure.*

#### **AREAS OF USE:**

*Hard surfaces in: Surgery, Recovery,  
 Anesthesia, X-Ray, Cath. Lab,  
 E. R., Orthopedics, Newborn Nursery,  
 Respiratory Therapy, Radiology,  
 Central Supply,  
 Ultrasonic Cleaners*

#### **ORDERING INFORMATION:**

*No. 7020 - 32 oz. Bottle Spray..... 2/case  
 No. 7021 - 1 Gallon..... 4/case  
 No. 7022 - 2.5 Gallon..... each  
 No. 7023 - 5 Gallon..... each  
 No. 7024 - 55 Gallon..... each*



**MADA, INC.**  
 625 Washington Avenue  
 Carlstadt, NJ 07072  
 Toll Free: 800-526-6370  
 Tel: 201-460-0454  
 Fax: 201-460-3509

See our Web Page: [www.madamedical.com](http://www.madamedical.com) for a listing of our products

MFD rev. 2/07

Distributed By: