

HGH THAILAND - FULL INSTRUCTIONS WITH PHOTO OF ACTIVATION GENOTROPIN GOQUICK 36IU, HOW TO MAKE HGH INJECTIONS AND WHAT SIZE OF NEEDLES



THE COURIER PACKAGE IN THAILAND CONTAINS:

- Genotropin Goquick Pen 12mg (36IU)
- Medical container with ice
- Needles BD Micro-Fine Ultra 8mm x 0,25mm (40 units for each pen)
- Alcohol swabs



HGH WHAT NEEDLES TO USE? WHAT SIZE OF HGH NEEDLES?



the small size of needles, the injection of HGH is completely painless





With the free delivery of HGH in Thailand for injection of Human Growth Hormone in the cooling box we put needles **BD Micro-Fine Ultra 8mm x 0,25mm** or any other manufacturer of the correct size (29-gauge, 30-gauge, or 31-gauge)

step-by-step instruction on the activation of Genotropin pen goquick 36IU Package Content Genotropin:



Important information! when the pen is activated, the pen must be in the vertical position with the needle top

HGH pen with ready-made injection solution

Instruction to the preparation

NOTE - Needles are not included in the composition, we provide free of charge in the kit



Next, we need to set up a needle, for Genotropin pens use needles

BD micro fine + 0.25 x 5-8mm (29,30 or 31 G)

Important information! when the pen is activated, the pen must be in the vertical position with the needle top

Needles for growth hormone are sterile, just remove the protective packaging



further simply we twist a small corner to the handle of a growth hormone

Now, HGH is ready for mixing



Important information! when the pen is activated, the pen must be in the vertical position with the needle top

For convenience, the upper cap can be temporarily removed



It is convenient to take the upper part of the Genotropin Pen with your left hand



Important information! when the pen is activated, the pen must be in the vertical position with the needle top

Now we begin to turn the pen of growth hormone with the needle clockwise, screwing it inside. The task from position **A** is to turn the knob through position **B** to the end to position **C**



Important information! when the pen is activated, the pen must be in the vertical position with the needle top

During the rotation, the growth hormone is completely displaced and ready for injection



You will receive 12 mg (36 IU) ready-to-use solution of growth hormone for injection

DOSAGE OF GROWTH HORMONE GENOTROPIN

now you need to set the dosage of growth hormone for injection

0.3 milligram HGH is equal to 1 IU

0.6 milligram HGH is equal to 2 IU

0.9 milligram HGH is equal to 3 IU etc

Just move the wheel in any direction up or down to set the dosage

now everything is ready for injection, about how to do an injection of growth hormone, please read in the next



HOW TO INDEPENDENTLY INJECT A GROWTH HORMONE?

After you <u>activated</u> the **HGH pen** set the dosage you need, you can start injecting



everything is ready for injection, the injection is done with clean hands and the skin is pre-treated with alcohol

The task of injecting growth hormone under the skin into the fat layer (not muscle)

Note! You can use any place on the body (buttock, leg, stomach or shoulder)

- Next, you need to enter a needle to a depth of 5 millimeters, you can do at a right angle or slope - as in the picture



Then just press the button and hold for 10 seconds, for complete absorption

Please do not worry, the needle is very thin, it does not hurt at all



Ready! it's very simple with the help of injections of growth hormone, you are getting younger and healthier every day!