

Evacuation Instructions

From Screen to Clean

The Two Step System Too Simple to Ignore



VACUUM SHOCK 1

MONTHLY PRE-CLEANING SYSTEM



Step 1 of a 2-part system

Monthly use in conjunction with weekly use of Vacuum Clean (step 2) is proven to keep vacuum lines fresh and functioning at peak efficiency.

Ref# 3546, 6 tablets/jar

- Restores vacuum pressure
- Continuous self-activating cleaner
- Extends the life of pump and vacuum lines
- Keeps high/low speed lines clean and fresh
- Neutral formulations with a pH of 7 *

To clean evacuation trap:

1. Locate evacuation trap in operatory, usually near the chair. Remove lid, insert new trap if needed.
2. Insert one Vacuum Shock tablet in each evacuation trap and replace lid.
3. Turn pump on and vacuum up approx. one gallon of warm/hot water through the speed lines.
4. Turn pump off and empty buildup in main pump filter after cleaning HVE line.

To clean cuspidor:

Vacuum Shock tablet may take 3-5 days to dissolve in the trap. This is normal. When tablet is completely dissolved, resume use of Vacuum Clean as directed.

VACUUM CLEAN 2

WEEKLY MAINTENANCE SYSTEM



Step 2 of a 2-part system

Weekly use of Vacuum Clean in conjunction with monthly use of Vacuum Shock (step 1) is proven to keep your lines fresh and functioning at peak efficiency.

Ref# 3547, 45 tablets/jar

- Restores vacuum pressure
- Continuous self-activating cleaner
- Extends the life of pump and vacuum lines
- Keeps high/low speed lines clear of debris & fresh smelling
- Neutral formulations with a pH of 7 *

To clean evacuation trap:

1. Turn vacuum pump off.
2. Locate evacuation trap in operatory, usually near the chair. Remove lid, insert new trap if needed.
3. Insert one Vacuum Clean tablet in evacuation trap and replace lid.
4. Turn pump on. Tablet is now activated.
5. Insert new tablet every 3-5 days, depending on usage.

To clean high/low speed lines:

Dissolve 5 Vacuum Clean tablets in Pour & Clean 16 oz. dispenser (#3547P) and add hot water. Two oz. concentrated solution equals 1 gal. antimicrobial rinse. Aspirate 1 qt. per operatory to treat HVE and low speed lines daily.

* EPA passed a rule specific to Best Management Practices for Dental Amalgam Waste, prohibiting the use of bleach or chlorine-containing cleaners that may lead to the dissolution of solid mercury when cleaning chair-side traps and vacuum lines. The rule says, "...vacuum lines that discharge amalgam process wastewater to a POTW [publicly owned treatment works] must not be cleaned with oxidizing or acidic cleaners, including but not limited to bleach, chlorine, iodine and peroxide that have a pH lower than 6 or greater than 8." (40 CFR 441.30(b)(2)). Source: epa.gov/eg/dental-effluent-guidelines.

Ultrasonic Instructions

ULTRA SUPER SONIC™

ULTRASONIC CLEANING AND INSTRUMENT PRESOAK TABLET



Ultrasonic Presoak Tablet

A technologically advanced ultrasonic detergent and instrument pre-cleaning tablet, ULTRA SUPER SONIC is chemically engineered to remove bacteria and bioburden, achieving maximum instrument cleaning quickly and efficiently. No mixing is required with self-dissolving tablets that save time, storage space, and shipping costs.


Ref# 3559, 30 tablets/jar

- Non-toxic
- Non-corrosive
- Easy to Use
- Eliminates pre-scrubbing
- FDA Registered
- Neutral formulations with a pH of 7 *

One tablet makes one gallon of ultrasonic solution.

1. Place tablet(s) in ultrasonic bath.
2. Fill with water and allow 5 minutes to dissolve.
3. Place instruments and/or cassettes in bath for recommended times based on ultrasonic manufacturer's instructions for use.
4. Rinse instruments with water. For best results, dry instruments prior to sterilization. Do not leave instruments in bath overnight.

Evacuation & Ultrasonic Product Line

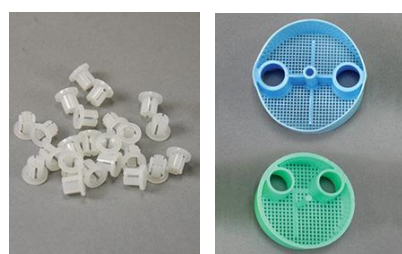

Ref#	Product	Quantity
3547S	E-VAC System Set™	2 jars/kit, 1 Vacuum Shock & 1 Vacuum Clean
3546	Vacuum Shock™	6 tablets/jar
3547	Vacuum Clean™	45 tablets/jar
3546K	Shock & Clean Starter Kit	1 kit, 1 Vacuum Shock, 1 Vacuum Clean & 1 (16 oz.) Pour and Clean Bottle
3547P	Pour and Clean Bottle	1 (16 oz.) bottle
3546O	Shock & Clean Operatory Kit	6 tubes/kit
3546TK	Shock & Clean Trail Kit	1 trial tube
1602 Series	Saliva Ejector Screens	25-Pack (1602), 50-Pack (1602A), 75-Pack (1602C), 100-Pack (1602B), 250-Pack (1620D)
1805	Evac Screens, Blue 2 1/8"	144 Pack, A-DEC with plastic canisters
1807	Evac Screens, Green 1 7/8"	144 Pack, Belmont, Biotec, Chayes, Dentech, DentalEZ, Pelton/Crane, Proma and others
3559	Ultra Super Sonic	30 tablets/jar

For more information, visit palmerohealth.com, call 800.344.6424 or email customerservice@palmerohealth.com.



Protecting Clinicians, Patients, and the Practice