

What are the instructions for use and cleaning of Spatulas?

Frequently Asked Questions

Directions for use:

1. Clean spatula before using with soap and warm water.
2. Use for mixing alginates, plaster, stone, investments, and other impression materials.

Directions for cleaning:

1. When done mixing, wipe excess material off with a damp cloth or moist towelettes. Hardened materials may be removed with a light solvent wipe such as Citrisol™ Orange Solvent Towelettes.
2. Clean remaining residue with soap and water.
3. Remove all traces of impression material before autoclaving.
4. All spatulas are autoclavable up to 270 degrees.

Ergonomic Spatula has a sturdy, high-grade polypropylene handle with flexible tip to optimize mixing ability. The spatula design is offered in various options which include change in size, color, scented or unscented, and left handed use.



Ref # 1521 W



Ref # 1520s

For more information, visit palmerohealth.com, call 800.344.6424 or email customerservice@palmerohealth.com.