


What are the instructions for use for Citrisol™ Orange Solvent Towelettes?

Frequently Asked Questions

Citrisol™ Orange Solvent Towelettes are premoistened orange solvent towelettes enhanced with natural oils and emollients. This allows them to quickly remove cements, adhesives, grease, gum, tar, show scuffs and a variety of other materials from almost any surface found in the dental operator or other health care environment. Lightly fragranced with a pleasant fresh citrus scent, these 9.5" x 12" towelettes are safe for hands and skin. These towelettes have a dual surface: one side is abrasive to loosen, the other is smooth to wipe clean, as indicated in the following figure.

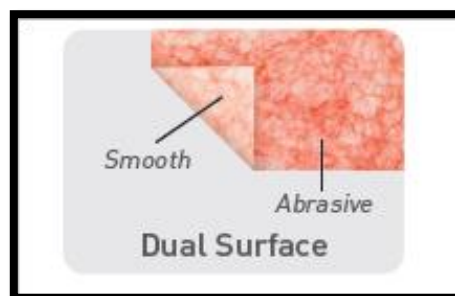

Directions for use:

1. Scrub using the abrasive side of towelette to loosen grease.
2. Use the smooth side of the towelette to wipe clean.

Directions for dispensing:

1. Remove large cover from container and discard seal.
2. From center of towelette roll, pull up a corner, twist into a point and thread through hole in container cover. Pull through approximately one inch.
3. Replace cover.
4. Pull out first towelette and snap off at a 90° angle.
5. Remaining towelettes will feed automatically, ready to use!
6. When not in use, keep center cap closed to prevent moisture loss.

For more information, visit palmerohealth.com, call 800.344.6424 or email customerservice@palmerohealth.com.



Protecting Clinicians, Patients, and the Practice