

Accelerate Operator Turnover with Confidence

Dental **PRODUCT
SHOPPER**

**RECOMMENDED
PRODUCT**

DisCide Ultra is a hospital-level, 1-step, ready-to-use quaternary ammonium, intermediate high-level alcohol-based disinfectant, developed to enhance clinical settings.



Contact Time

1-step cleaner and disinfectant with rapid **1 minute** contact time, specifically formulated to achieve required contact time.



Kill Claims

In just 1 minute or less, DisCide Ultra is effective against TB, viruses (HBV, HCV, RSV, HIV-1), bacteria (MRSA, VRE, E. coli), and fungi (C. albicans).



Chemical Formula

Proprietary formulation is non-corrosive, non-staining, and leaves no residue with a pleasant herbal scent.



Convenient

Identical formulations available in both spray and towelette formats to ensure efficacy and ease of use when liquid and towelettes are used in tandem. Non-woven, lint-free, and durable towelettes are premoistened and ready-to-use.



Compliance

Meets CDC, OSHA, and infection control guidelines. EPA and FDA registered.

*Treated surface must remain visibly wet for 1 minute

DisCide® ULTRA

Palmero is now a proud member of HuFriedyGroup

PALMERO®
HEALTHCARE
Protecting Clinicians, Patients, and the Practice

DisCide[®] ULTRA

Instructions for Use:

If less than 5% bioburden present, 1-step cleaning and disinfecting can occur.

CLEANING before disinfecting

- Step 1: Spray hard non-porous surface with DisCide Ultra Spray to help loosen bioburden.
- Step 2: Wipe down the surface with a DisCide Ultra towelette to remove debris and bioburden.
- Step 3: Dispose of used DisCide Ultra towelette.

DISINFECTING after cleaning

- Step 1: Use DisCide Ultra towelette or Spray DisCide Ultra Spray on the pre-cleaned surface.
- Step 2: Leave the surface wet for one minute.
- Step 3: Wipe down the surface with a DisCide Ultra towelette to remove any residue chemical left behind.
- Step 4: Dispose of used DisCide Ultra towelette.

Disinfectant Chart:

MICROORGANISM TESTED		DISCIDE ULTRA TOWELETTES	DISCIDE ULTRA LIQUIDS
EPA REGISTRATION NUMBER		10492-4	10492-5
GRAM-POSITIVE BACTERIA	MRSA-METHICILLIN-RESISTANT STAPHYLOCOCCUS AUREUS	1 MIN	1 MIN
	STAPH-STAPHYLOCOCCUS AUREUS	1 MIN	1 MIN
	VRE-VANCOMYCIN-RESISTANT ENTEROCOCCUS FAECALIS	1 MIN	
	VRE-VANCOMYCIN-RESISTANT ENTEROCOCCUS FAECIUM		1 MIN
GRAM-NEGATIVE BACTERIA	ANCINETOBACTER BAUMANNII		1 MIN
	BURKHOLDERIA CEPACIA	1 MIN	1 MIN
	CAMPYLOBACTER JEJUNI	1 MIN	1 MIN
	CARBAPENEM-RESISTANT E COLI		1 MIN
	E COLI-ESCHERICHIA COLI	1 MIN	1 MIN
	ESCHERICHIA COLI O157:H7	1 MIN	1 MIN
	KLEBSIELLA PNEUMONIAE		1 MIN
	PROTEUS MIRABILIS	1 MIN	1 MIN
	PSEUDOMONAS AERUGINOSA	1 MIN	1 MIN
	SALMONELLA ENTERICA	1 MIN	1 MIN
ENVELOPED VIRUSES	AVIAN INFLUENZA A VIRUS-H3N2	30 SEC	30 SEC
	H1N1-PANDEMIC 2003 INFLUENZA A VIRUS (FORMERLY SWINE FLU)	30 SEC	30 SEC
	HBV-HEPATITIS B VIRUS	1 MIN	1 MIN
	HCV-HEPATITIS C VIRUS	1 MIN	1 MIN
	HERPES SIMPLEX VIRUS TYPE 2	1 MIN	1 MIN
	HIV-1 (AIDS VIRUS)	1 MIN	1 MIN
	HUMAN CORONAVIRUS	30 SEC	30 SEC
	INFLUENZA A VIRUS (HONG KONG)	30 SEC	30 SEC
	RSV-RESPIRATORY SYNCYTIAL VIRUS	1 MIN	1 MIN
	VACCINIA VIRUS	1 MIN	1 MIN
NON-ENVELOPED VIRUSES	ADENOVIRUS TYPE 2	1 MIN	1 MIN
FUNGI	CANDIDA ALBICANS	1 MIN	1 MIN
MYCOBACTERIA	TB-MYCOBACTERIUM BOVIS (TUBERCULOSIS)	1 MIN	1 MIN
TOTAL MICROORGANISMS TESTED		23	26



Ordering Information:

TOWELETTES



Ref# 10DIS
10" x 10" towelettes
60 count canister



Ref# 60DIS
6" x 6 3/4" towelettes
160 count canister

SPRAYS



Ref# 3565Q
Quart Sprayer



Ref# 3564
4 oz. Bottle

REFILLS



Ref# 3565G
Gallon Refill

ACCESSORIES



Ref# 3567
Empty quart bottle with sprayer and DisCide ULTRA label



Ref# 3567T
Quart spray trigger



Ref# 3519
Unlabeled empty quart bottle with sprayer

For more information, visit palmerohealth.com, email customerservice@palmerohealth.com or call 800.344.6424.

Palmero is now a proud member of HuFriedyGroup

