

# DisCide<sup>®</sup> ULTRA

## One Minute Kill. Accelerates Operatory Turnover.



### Infection Control Lineup Offers Fast-Acting Asepsis Solutions

DisCide Ultra is a hospital-level, one-step, ready-to-use quaternary ammonium, intermediate high-level alcohol-based disinfectant. With this intermediate level surface disinfectant and decontaminant cleaner, you can accelerate the way you disinfect clinical settings.

### Zero to Gone In 60 Seconds

In just 60 seconds, DisCide Ultra is effective against TB, viruses (HBV, HCV, RSV, HIV-1), bacteria (MRSA, VRE, E. coli), and fungi (C. albicans).



### Proprietary Formulation

Specifically formulated to resist evaporation, it's non-corrosive, non-staining, and leaves no residue with a pleasant herbal scent.

### Clinically Proven

Proven to kill deadly pathogens in one minute or less for accelerated operatory turnover. EPA and FDA registered.

### Standard Compliance

Listed with the United States Food and Drug Administration and registered with the Environmental Protection Agency (EPA), DisCide Ultra meets disinfection requirements of the federal Occupational Safety and Health Administration's (OSHA) Bloodborne Pathogens Standard.

### Convenient Packaging

Pre-mixed for the ultimate in convenience. Available in identical formulations in both spray and towelette formats to ensure efficacy when liquid and towelettes are used in tandem.

# DisCide<sup>®</sup> ULTRA



## Instructions for Use:

*If less than 5% bioburden present, 1-step cleaning and disinfecting can occur.*

### CLEANING before disinfecting

- Step 1:** Spray hard non-porous surface with DisCide Ultra Spray to help loosen bioburden.
- Step 2:** Wipe down the surface with a DisCide Ultra towelette to remove debris and bioburden.
- Step 3:** Dispose of used DisCide Ultra towelette.

### DISINFECTING after cleaning

- Step 1:** Use DisCide Ultra towelette or Spray DisCide Ultra Spray on the pre-cleaned surface.
- Step 2:** Leave the surface wet for one minute.
- Step 3:** Wipe down the surface with a DisCide Ultra towelette to remove any residue chemical left behind.
- Step 4:** Dispose of used DisCide Ultra towelette.

## Disinfectant Chart

MICROORGANISM TESTED	DISCIDE ULTRA TOWELETTES	DISCIDE ULTRA LIQUIDS
EPA REGISTRATION NUMBER	10492-4	10492-5
ADENOVIRUS TYPE 2	1 MIN	1 MIN
ANCINETOBACTER BAUMANNII - <i>NEW Claim</i>		1 MIN
AVIAN INFLUENZA A VIRUS-H3N2	30 SEC	30 SEC
BURKHOLDERIA CEPACIA - <i>NEW Claim</i>	1 MIN	1 MIN
CAMPYLOBACTER JEJUNI - <i>NEW Claim</i>	1 MIN	1 MIN
CANDIDA ALBICANS	1 MIN	1 MIN
CARBAPENEM-RESISTANT E COLI		1 MIN
E COLI-ESCHERICHIA COLI	1 MIN	1 MIN
ESCHERICHIA COLI O157:H7 - <i>NEW Claim</i>	1 MIN	1 MIN
H1N1-PANDEMIC 2003 INFLUENZA A VIRUS (FORMERLY SWINE FLU)	30 SEC	30 SEC
HBV-HEPATITIS B VIRUS	1 MIN	1 MIN
HCV-HEPATITIS C VIRUS	1 MIN	1 MIN
HERPES SIMPLEX VIRUS TYPE 2	1 MIN	1 MIN
HIV-1 AIDS VIRUS	1 MIN	1 MIN
HUMAN CORONAVIRUS	30 SEC	30 SEC
INFLUENZA A VIRUS (HONG KONG)	30 SEC	30 SEC
KLEBSIELLA PNEUMONIAE - <i>NEW Claim</i>		1 MIN
MRSA-METHICILLIN-RESISTANT STAPHYLOCOCCUS AUREUS	1 MIN	1 MIN
PROTEUS MIRABILIS	1 MIN	1 MIN
PSEUDOMONAS AERUGINOSA	1 MIN	1 MIN
RSV-RESPIRATORY SYNCYTIAL	1 MIN	1 MIN
SALMONELLA ENTERICA	1 MIN	1 MIN
STAPH-STAPHYLOCOCCUS AUREUS	1 MIN	1 MIN
TB-MYCOBACTERIUM BOVIS (TUBERCULOSIS)	1 MIN	1 MIN
TRICHOPHYTON MENTAGOPHYTES-ATHLETE'S FOOT FUNGUS	1 MIN	1 MIN
VACCINIA VIRUS - <i>NEW Claim</i>	1 MIN	1 MIN
VRE-VANCOMYCIN-RESISTANT ENTEROCOCCUS FAECALIS	1 MIN	
VRE-VANCOMYCIN-RESISTANT ENTEROCOCCUS FAECIUM		1 MIN
<b>TOTAL MICROORGANISMS TESTED</b>	<b>24</b>	<b>27</b>

## Ordering Information:

### TOWELETTES



Ref# **10DIS**  
10 1/2" x 10" towelettes  
60 count canister

Ref# **60DIS**  
6" x 6 3/4" towelettes  
160 count canister

### SPRAYS



Ref# **3565Q**  
Quart Sprayer

Ref# **3564**  
4 oz. Bottle

### REFILLS



Ref# **3565G**  
Gallon Refill

### ACCESSORIES



Ref# **3567**  
Empty quart bottle  
with sprayer and  
DisCide ULTRA label

Ref# **3567T**  
Quart spray trigger

Ref# **3519**  
Unlabeled empty  
quart bottle with  
sprayer



For more information, visit [palmerohealth.com](http://palmerohealth.com), email [customerservice@palmerohealth.com](mailto:customerservice@palmerohealth.com) or call 800.344.6424.