

Quick Start Guide




BEHIND THE EAR
OTC HEARING AIDS



For detailed information on how to use your BTEs, please refer to the User Manual.

BTE4 HEARING AIDS AND CHARGING CASE

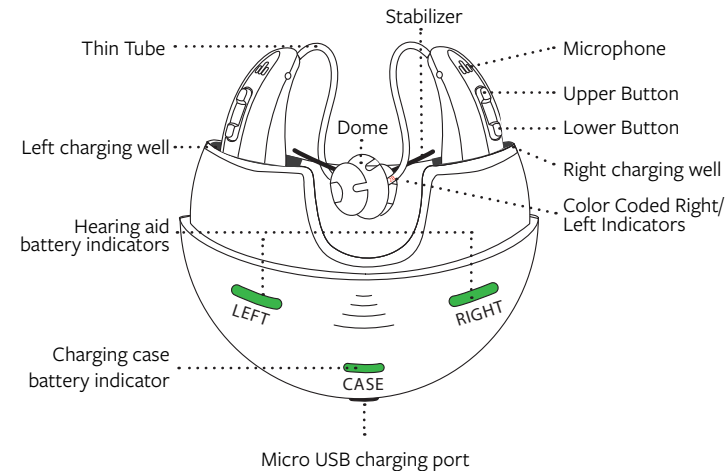

STEP ONE

Charge Your Hearing Aids

Hearing aids should be fully charged before using for the first time.


Indicator Light on Charging Case

Orange lights	Hearing aid is charging
Green lights	Hearing aid is fully charged

It will take approximately 3.5 hours to fully charge hearing aids.

STEP TWO

Insert Hearing Aids

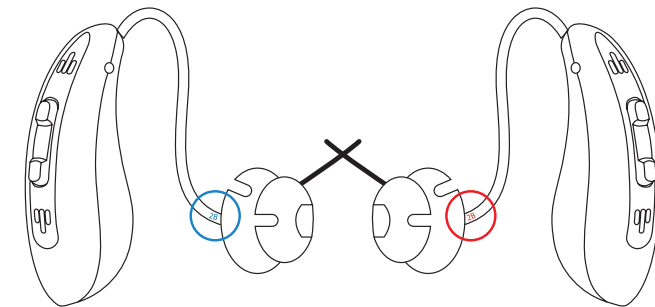


Blue indicator for left ear.


Red indicator for right ear.

STEP TWO, CONT...

Hold the thin tube bend with your thumb and finger as shown. Gently insert the dome into your ear canal.

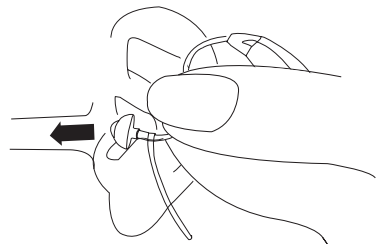


Using your finger, position the stabilizer into the curvature of your ear. The stabilizer helps keep the aid secure in your ear.

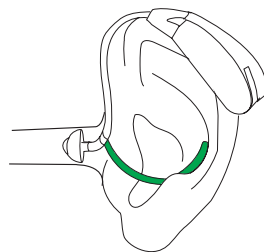

STEP TWO, CONT...

Place hearing aid behind your ear.

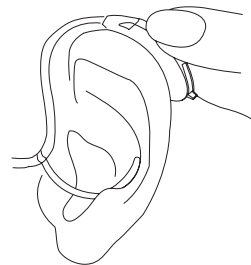

Turn hearing aid on.

If hearing aid is off, press and hold the lower button for 3 seconds until the hearing aid turns on.

STEP THREE

Make Adjustments to Your Hearing Aid Using the Upper and Lower Button


UPPER BUTTON



Press & Hold
CHANGE PROGRAM



Quick Press
VOLUME INCREASE

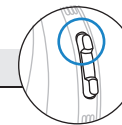


LOWER BUTTON



Press & Hold
TURN ON & OFF



Quick Press
VOLUME DECREASE

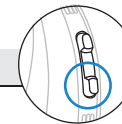

STEP FOUR

Remove Hearing Aid

Grasp thin tube bend at the entrance of the ear canal. Gently pull thin tube out of ear, removing dome from ear.

Remove hearing aid from behind the ear.

Wipe dome clean from any wax, oils, moisture, etc.

Power aid off. Place aid in charging case when not in use.



Tip: To avoid feedback/chirping when aid is being removed, turn aid off prior to removal.