PRODUCT SPECIFICATION

Product Range:		AZOWIPE™	
Product Code:	Product Description:	Substrate:	Dimensions W & L plus Tolerances:
81103	AZOWIPE™ 70% IPA BP disinfectant Wipes for rapid and effective disinfection.	Thermo-bonded viscose (60%) and polypropylene (40%)	200mm ± 10mm x 200mm ± 5mm
Composition		70% V/V solution of Isopropanol BP (propan-2-ol) blended with 30% deionised water, filtered to 0.2 micron.	
pH Information:		Range 6.0 - 7.0	
Intended Use:		Intended to be used on hard surfaces as a disinfectant wipe.	
Classification:		Biocide.	
Product/Device single use:		Yes - individual wipes are single use.	
Components latex free:		Yes.	
Shelf life:		36 months from date of manufacture.	
Storage:		Store upright away from direct sun light and sources of heat and ignition. Keep container closed between uses.	
Method of sterilization:		Non-sterile.	
Special Instructions:			
Refer to SDS for safety data. Refer to canister artwork for opening instructions. Do not mix with other chemicals. Do not use the same wipe on more than one surface. Alcohol wipes should not be used on shellac mountings of lensed instruments or plastic tubing.			
Case Quantity Case Weight:		Primary Packaging:	Secondary Packaging:
Per Container: 200 wipes Per Case: 12 canisters Case Weight: 9.11kg		White high density polyethylene canister, PET metal foil seal and blue polypropylene lid with an aperture of 6mm for wipe dispensing. Canister dimensions 110 x 240mm.	Cardboard case. Case dimensions: 445 x 330 x 260mm
Pallet Dimensions:		Pallet Type:	Pallet Configuration:
1200mm x 1000mm		4-WAY	Cases per layer: 6 cases Layers per pallet: 5 layers Quantity per pallet: 30 cases
Pallet Weight:			
275kg approx			
Standards of Compliance:			
IPA solution is manufactured in accordance with the British Pharmacopoeia. EU Biocide Regulation EU 528/2012. EN1276. EN13727. EN14348. EN14476. EN1650. EN13624.			

Labelling specification :

Primary Label (SHI4831):

EN Caution: H205 Highly flammable, H219 Causes serious eye irritation, H302 May cause drowsiness or dizziness. Storage: Keep container tightly closed. Keep upright in a cool dry place, away from heat and sources of ignition. For professional use only.

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REF: 81103
SHI 4831

SYNERGY HEALTH
Synergy Health PLC, 11th Floor, 100 Broad Street, London, W1P 3JF, United Kingdom
AZOWIPE is a trademark of Synergy Health Ltd

AZOWIPE™
Disinfectant wipes
For effective and rapid disinfection
200 Wipes

Type	Test method	Microorganism	Contact time	Testing conditions
Viruslike	EN 14476	BVDV, HEN, HDV	15 sec	Clean
Viruslike	EN 14476	Vaccinia, H1N1	15 sec	Clean
Bakterieel	EN 13727	Mycobacterium terrae TB	30 sec	Clean
Bakterieel	EN 13727	Protus mirabilis	30 sec	Clean
Bakterieel	EN 13727	Pseudomonas aeruginosa	1 min	Dirty/Clean
Bakterieel	EN 13727	Staphylococcus aureus	1 min	Dirty/Clean
Bacterieel	EN 13727	Enterococcus faecalis	1 min	Dirty/Clean
Bacterieel	EN 1279	E.coli	1 min	Dirty/Clean
Bacterieel	EN 13727	MFC	1 min	Dirty/Clean
Fungicide	EN 13624	Candida albicans	2 min	Dirty/Clean
Fungicide	EN 13624	Aspergillus niger	2 min	Clean

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Secondary Label (shipper) (SHI5329):

Synergy's name and address and symbol. Product name. Product REF. Quantity. Product code. LOT number and symbol. Expiry Date and symbol. Flammable symbol. Irritant Symbol. Barcode containing (240) product reference (10) LOT number (17) expiry date. ↑↑ This way up symbol.

Synergy Health (UK) Limited Approval:	
Quality:	
Name: Mina Patel	Signed:
Position: Principal QA/RA Officer	Date: 06/08/2014
Marketing:	
Name: Joanna Silva	Signed:
Position: Senior Product Manager	Date: 06/08/2014