

Three Year 100% Performance Guaranty / Warranty

Perspectis, Inc. ("Perspectis") warrants, subject to the conditions set forth below, that should this product be defective due to poor workmanship or materials any time during the specified warranty period, Perspectis will repair or replace the same with the latest model, whatever is fair, without charge for either parts or labour. Shipping charges may apply.

The warranty period is THREE full years from the date of original purchase at retail.

CONDITIONS

1. **Registration:** The attached Warranty Registration Form must be submitted to Perspectis within forty five (45) days after the date of original purchase.
2. **Proof of Date of Purchase:** The owner must furnish proof of original purchase, should the Warranty Registration Form not be returned.
3. **Unauthorized repair, abuse, etc.:** The unit must not have been altered, modified, or repaired. The unit must not have been subject to accident, misuse, abuse, commercial use, rental, or operated contrary to the User Instructions.
4. This warranty does not extend to normal wear and tear and damage caused by misuse or improper care, does not cover regular product maintenance such as cleaning or lubrication, and is valid only within the country where the unit is originally purchased.
5. **Soapy-water bubble test** (apply soapy water on suspected leaking spot of an inflated Lumbar Decompressor and press on it. A bubble formation indicates a leakage; no bubble, no leakage. Caution: Don't let the monitor get wet, in case your device has a pressure monitor.) needs to be performed by the owner before a warranty service inquiry is submitted.
6. **Proper delivery:** The owner must call or email Perspectis for authorization (RMA #) and instructions prior to returning the defective product for warranty inspection. Any unauthorized returns will be refused or shipped back at the owner's expenses. Upon receiving Perspectis' warranty authorization and instructions, the defective unit must be shipped, freight prepaid, to the designated Perspectis facility in either its original package or similar package affording an equal level of protection. The owner must include the RMA # with the shipping address; and a note with their name, address and telephone number along with a description of the defect.

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, PERSPECTIS, INC. EXPRESSLY DISCLAIM ALL OTHER WARRANTIES AND CONDITIONS, EXPRESS OR IMPLIED, AND WHETHER ARISING BY LAW, BY STATUTE, BY COURSE OF DEALING OR USAGE OF TRADE, INCLUDING WITHOUT LIMITATION IMPLIED WARRANTIES OR CONDITIONS OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. UNDER NO CIRCUMSTANCE SHALL PERSPECTIS, INC. OR ITS SUPPLIERS, VENDORS OR DISTRIBUTORS BE LIABLE FOR AN AMOUNT GREATER THAN THE ACTUAL PURCHASE PRICE OF THE UNIT OR FOR ANY DIRECT, INDIRECT, SPECIAL, PUNITIVE, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, OR ANY DAMAGE WHATSOEVER, RESULTING FROM ANY USE OF THIS PRODUCT, OR ANY OTHER CAUSES, WHETHER BASED ON PRODUCT LIABILITY, OR OTHERWISE, ARISING OUT OF OR IN ANY WAY CONNECTED WITH THE USE OR PERFORMANCE OF THIS PRODUCT, WITH THE INABILITY TO USE THIS PRODUCT, OR WITH THE PROVISION OF OR FAILURE TO MAKE AVAILABLE ANY OF ITS OR THEIR PRODUCTS, GOODS, OR SERVICES, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. AND PERSPECTIS, INC. NEITHER ASSUMES NOR AUTHORIZES ANY REPRESENTATIVE OR OTHER PERSON TO ASSUME FOR IT AN OBLIGATION OR LIABILITY OTHER THAN AS IS EXPRESSLY SET FORTH HEREIN. THIS PRODUCT IS OFFERED TO YOU CONDITIONAL UPON YOUR ACCEPTANCE WITHOUT MODIFICATION OF THE LIMITATIONS SET FORTH ABOVE. TEST AND/OR USE OF THIS PRODUCT IMPLIES ACCEPTANCE OF ALL SUCH LIMITATIONS.



LUMBAR DECOMPRESSOR™

User Instructions & Three Year 100% Performance Guaranty / Warranty (must read first)



Customers located in USA and Canada:

Please **DO NOT** contact place of purchase for warranty assistance.

Call toll free 1-855-BodyRyzm for any warranty questions or services.

BodyRyzm LifeSciences / Perspectis, Inc.

1 First Canadian Place, Suite 350

Toronto, ON, Canada, M5X 1C1

www.BodyRyzm.com

Toll free: 1-855-BodyRyzm (263-9799), Tel: 1-416-595-1575

Fax: 1-416-595-6438

Email: CustomerService@BodyRyzm.com



WARNING



- This product is not a toy. All users need to read and understand all instructions in this document before using or testing this product.
- Use lower inflation/pressure, if you are not sure! In no event should you feel uncomfortable when using the device.
- Start with zero inflation/pressure when you begin to use the device each time. Then gradually increase the level of inflation/pressure. Stop increasing inflation/pressure immediately once you feel that your back pain caused by nerve-compression has dissipated.
- Reduce inflation/pressure immediately, should you experience any discomfort.
- Do not use this product or do any exercise on or with this product, if you have acute back injury, spinal fusion, recent surgery, pregnancy or any other unusual condition.
- Consult licensed and qualified health advisor in case of any question or concern.
- Do not jump on this product.
- Do not sit or step on this product.
- Use this product across the small of your back only, when lying on it.
- This product is designed for weight loads up to 350 lbs. only.
- Keep this product away from any sharp objects, any aggressive or caustic chemicals, and any intense heat source.
- Adjust the inflation of this product, only when it is at room temperature.
- Keep the packaging bag away from babies and children to avoid danger of suffocation.


HOW TO ADJUST ...

You can easily and instantly adjust the level of inflation/pressure of your Lumbar Decompressor

- Increasing inflation/pressure:** turn the valve knob clockwise to the end firmly. Squeeze the inflation bulb repeatedly to inflate. Stop increasing inflation/pressure immediately once you feel that your back pain caused by nerve-compression has dissipated.

Depending on the specific model that you purchased, your device may have bulb pump only, or may have bulb pump with pressure monitor, as shown in the picture on the right.

- Decreasing inflation/pressure:** turn the valve knob counter-clockwise. Then press on the air cushion to deflate.

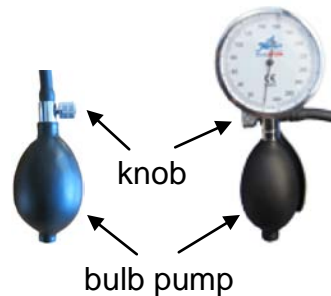

HOW TO ENJOY...

Reduce back pain by decompressing lumbar spine

Decompress Your Lower Back



- Place your Lumbar Decompressor flat on a flat surface such as bed, floor, sofa, etc.
- Lie gently on your Lumbar Decompressor. Make sure that your Lumbar Decompressor is centered across the small of your lower back.
- Turn the valve knob clockwise to the end firmly. Squeeze the inflation bulb pump repeatedly to pump air into your Lumbar Decompressor.
- Stop pumping immediately once you feel that your back pain caused by nerve-compression has dissipated.
- Continue lying on your Lumbar Decompressor for 30 - 60 min. or until you feel comfortable to get up, whichever is sooner.
- While lying on your Lumbar Decompressor, you may repeat deflating and re-inflating it intermittently, for better results.
- While lying on your Lumbar Decompressor, you may choose to bend your legs with your feet resting on the surface, for more comfort.

Reduce back pain by supporting your lower back

Support Your Lower Back



- Place your Lumbar Decompressor behind your back in bed, sofa or chairs.
- Lean or lie gently on your Lumbar Decompressor. Make sure that your Lumbar Decompressor is centered across the small of your lower back.
- Turn the valve knob clockwise to the end firmly. Squeeze the inflation bulb pump repeatedly to pump air into your Lumbar Decompressor.
- Stop pumping immediately once you feel that the support is sufficient or that your back pain caused by nerve-compression has dissipated.
- Continue leaning or lying on your Lumbar Decompressor for soothing back support.

HOW TO CARE FOR AND TROUBLESHOOT...

Simple care:

- Clean Lumbar Decompressor by wiping it with soft cloth or warm soapy water whenever necessary.

Simple trouble shooting:

- Turn the valve knob clockwise firmly to prevent pressure loss when inflating the device
- Soapy-water bubble test:** apply soapy water on suspected leaking spot of an inflated Lumbar Decompressor and press on it. A bubble formation indicates a leakage; no bubble, no leakage. Contact us within warranty period.