

ANTI-AGEING & SKIN REJUVENATION

The ultimate professional tool designed for the beauty, medical and aesthetic world. The frozen, medical grade surgical steel Cryoglobes are used when frozen for an invigorating facial and body massage, which cools skin. The Cryoglobes remain cold for longer due to their design. The Cryoglobes cause vasoconstriction and excites cryo-microcirculation. While beneficial on its own, this unique effect also allows other skincare products to better penetrate and rejuvenate your skin, working alongside other devices and treatments.

- Appearance of wrinkles and fine lines are reduced.
- Production of collagen resulting in fuller, more youthful and beautiful rejuvenated skin.
- Puffiness around the eyes is significantly reduced or totally eliminated
- Pores on the face and neck are reduced in size, limiting the intake of toxins, dirt and grime.
- Benefits the lymphatic system, flushing away built up toxins, soothes inflammation and irritation, removes puffiness and redness.
- Skin conditions like Dermatitis, Eczema, Acne and Psoriasis can be relieved.
- Metabolic rate increases. Endorphin levels are stimulated. After one session you can see the reduction of lines, puffiness and will experience a clearer skin colour.

The Cryoskin Cryoglobe is ideal to:

- Reduce inflammation
- Boost cryostimulation of the skin
- Removes toxins
- Improve & enhance circulation to the skin's surface
- Smooths and tighten the skin
- Minimise the appearance of large pores
- Reduce facial redness
- Smooth out fine lines and wrinkles
- Use as an invigorating ice facial, lymphatic massage
- Improve circulation in the skin – leaving a healthy glow

Use with the following treatments:

- Semi-permanent makeup
- Coolifting gun
- Vein removal
- Skin needling
- Injectables
- Face peels
- Radio frequency
- Microdermabrasion
- IPL
- Waxing
- Laser
- Dermaplaning
- Facials
- Massages

INSTRUCTIONS OF USE

Preparation:

- Place your Cryoglobe in the freezer in the cotton bag supplied and ensure that you freeze for 24 hours before performing your first treatment.
- To ensure best application, the Cryoglobe liquid must be frozen before use.
- Please note: freezer temperature should be as cold as you can make it.
- When frozen, remove the Cryoglobe from the freezer, roll Cryoglobe on a clean cloth or towel, removing any excess water or ice residue.

Application:

- Begin massage by gently rolling the Cryoglobe on target tissues for 5-8 minutes. Roll slowly and steadily, in circular, upward motions using an upward 'J' motion. Do not drag the skin. Keep the ball moving continuously.
- Use two Cryoglobes, in synergy.

Cleaning:

- Remove the metal globe from the holder handle. Sterilise with antibacterial wipes or under hot water and allow to dry separately.
- Place the handle and Cryoglobe back together and return them in the cotton bag provided to the freezer.

Warning:

- Do not open the sealed Cryoglobe or tamper with the screw
- Do not force when attaching the Cryoglobe to the handle, they are designed to fit together with ease
- Cryoglobes are only for external use.
- The Cryoglobe must be used in a continuous motion and we advise, not to rest it on any part of the skin.
- If interfered with in any way, this will invalidate your warranty.
- Do not autoclave.
- Do not heat up.
- Do not consume the fluid/liquid inside the Cryoglobe.
- If you have cold intolerance or allergic to stainless steel metal, please do not use the Cryoglobe.



Do not hold the Cryoglobe too long in on one area. Keep it moving.

