


COVID-19

Rapid SARS-CoV-2 Antigen Test Card

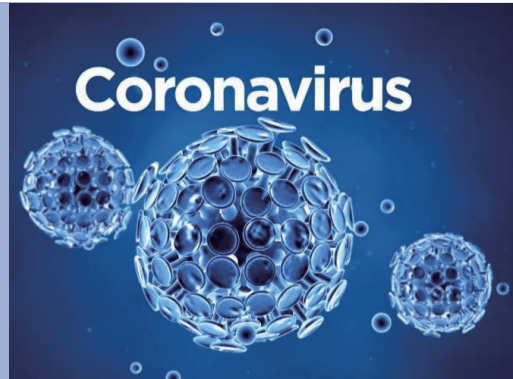

Using a nasal swab to get sample fluid, antigen tests can produce results in minutes. Because these tests are faster and less expensive than molecular tests are, experts consider antigen tests more practical to use for large numbers of people. Since the antigen test can be high specificity, a positive antigen test result is considered very accurate.



- **Fast, Easy and Convenient**

Results in 15-20 minutes, simple to use
Requires no instruments and special skills

- **Storage**

Room temperature storage & 18-month shelf life

- **Swabs Test**

Nasopharyngeal swab can be used

Cat. No.	Product Name	No. per Kit
1N40C5	Rapid SARS-CoV-2 Antigen Test Card	20
1N40C5-2	Rapid SARS-CoV-2 Antigen Test Card	1

Test Procedure

1
Add 8 drops of sample buffer (about 0.25 mL)

2
Insert swab, roll and mix

3
pinch the tube and remove the solution from the swab

4
Install the nozzle onto the tube

5
Add 3 drops of specimen and wait for 15-20 mins

Interpretation of Results

SARS-CoV-2 Antigen
ID _____

C T

Positive

SARS-CoV-2 Antigen
ID _____

C T

Negative

SARS-CoV-2 Antigen
ID _____

SARS-CoV-2 Antigen
ID _____

C T

C T

Invalid

BOSON