


ISSUE DATE:	18-JAN-2021	REVISION:	001	DOCUMENT NO.	MA-SPEC-3B
-------------	-------------	-----------	-----	--------------	------------

### AMAX LEVEL-3 (BLUE) PROCEDURAL FACE MASK SPECIFICATION

**Product Code:** AMSCL3B

**LOT#:** \_\_\_\_\_

**Description:** 6.9" x 3.75" (Level 3 Mask Blue with Aluminum Wire and Ear-Loops)

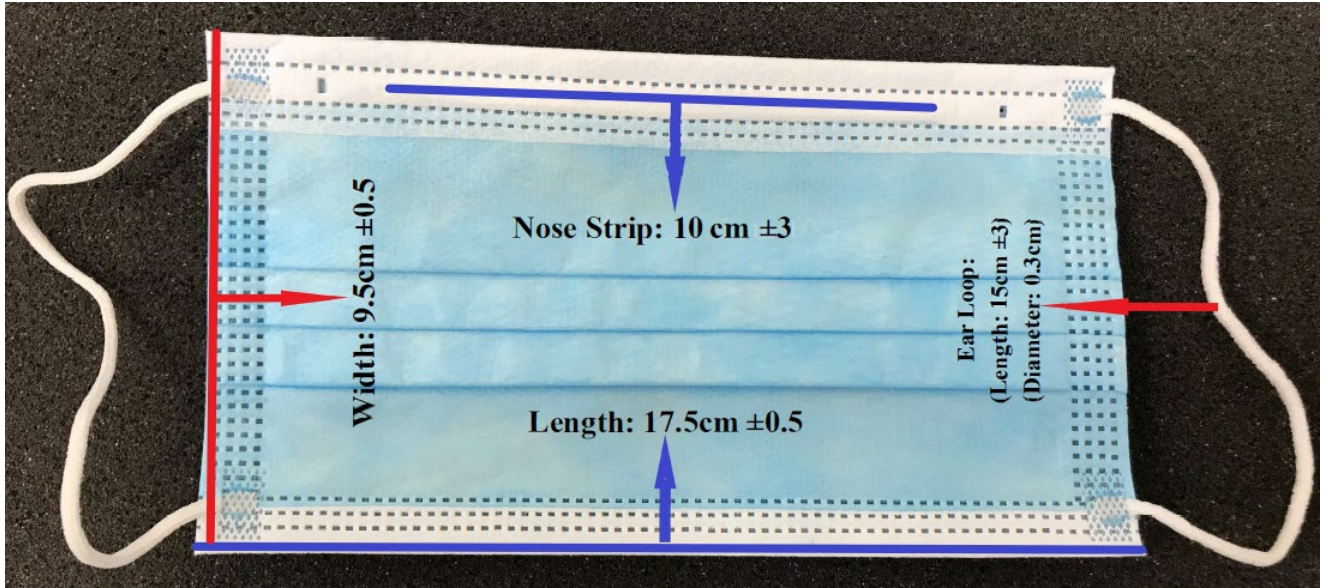

GS Medical Confidential

Item No.	Body Size		Inner Layer Spun-bond non-woven fabric Grade SSS White 25 gsm	Middle Layer Melt-blown non-woven fabric High Filtration White 20 gsm	Outer Layer Spun-bond non-woven fabric Grade SSS Blue 25 gsm	Nose Strip Aluminum Wire 10cm ±3	Ear Loop Elastic Polyester and Spandex Length: 15cm ±3 Diameter: 0.3cm
	Length	Width					
AMSCL3B	17.5cm ±0.5	9.5cm ±0.5					

#### QC Use Only

Average face mask ear loop tensile strength must be 10N minimum.

Label Issuance and Reconciliation				
Label Type	Quantity	Date of Issue	Quantity Remaining	Disposal

#### Production Summary

Production Start Date: \_\_\_\_\_

Production Finish Date: \_\_\_\_\_

Total Pallets: \_\_\_\_\_

Total Cases: \_\_\_\_\_

Total Units: \_\_\_\_\_

#### Operator and Packers Use Only

Line set-up and clearance sign off: \_\_\_\_\_ Date: \_\_\_\_\_

Packer sign off on line clearance: \_\_\_\_\_ Date: \_\_\_\_\_

#### Information check for Inner and Outer labels (Packer Sign off)

I have read the label and all information pertaining to this product is correct.

Signature: \_\_\_\_\_ Date: \_\_\_\_\_

**Inspection Components:**

- Check Mask quality of 3 random mask samples every 15 minutes.
- Visually inspect fabrics ultrasonic welds with no loose end.
- Ensure mask ends are neatly trimmed and Ear Loops are firmly attached.

Approved by (QC): \_\_\_\_\_ Date: \_\_\_\_\_