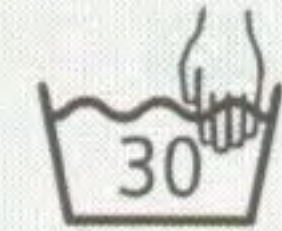
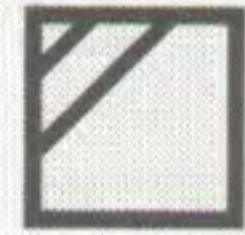
PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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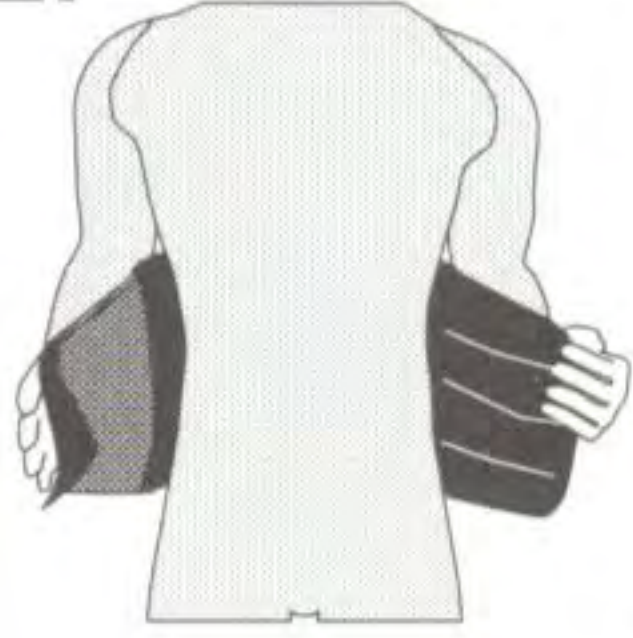
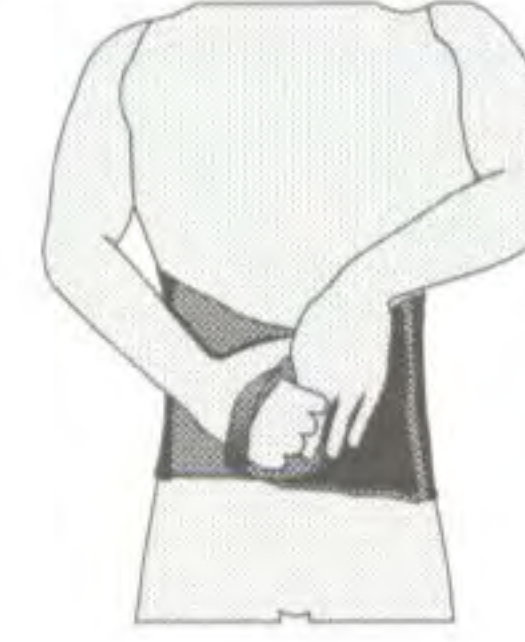
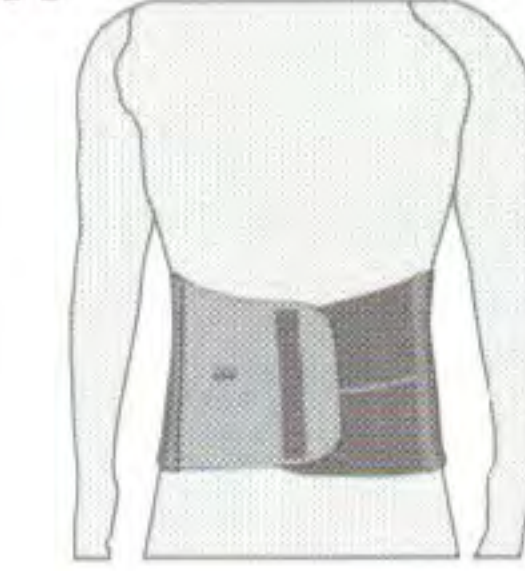
English

INTENDED USE :

Abdominal support is a compression belt to support the abdomen and compress the abdominal muscles. Its used as a post operative aid, by mothers after delivery and for waist line reduction.

CONSTRUCTION:

Made from extra strong, single wide panel of elastic containing heat resistant rubber with high modulus of elasticity. Broad hook loop closure provide perfect fitting of the belt around the waist, controlled compression, its quick, easy application and removal. Flexi Nylon splints ensure no wrinkles, no buckling and no rolling over of the belt. It doesn't show up from the clothes because of its sleek construction.

APPLICATIONS:

- To protect and bind surgical incisions of the abdomen and the chest.
- Postoperative care of thorax and abdomen.
- Reduce the discomfort of sudden rib cage expansion from deep breaths, cough or sneezing during recovery.
- For post delivery mothers, to tone up the mellowed abdominal muscles and set the waistline to its Pre-pregnancy dimensions.
- Used as a general tummy trimmer.
- In umbilical and ventral hernia.

RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
- Consult your doctor in case of concurrent use of another medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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Tynor Orthotics Pvt. Ltd.