

PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not wring the product



Do not dry-clean.



Do not Iron.



Do not bleach.



Close hook loop fasteners before washing.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :



Batch code



Manufacturing date



Manufacturer



Catalogue number

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001



W.H.O.
G.M.P.

ISO
13485



US FDA
registered

Abs Wrap (Neoprene)

ऐब्स रैप (नियोप्रेने)

Faja Abdominal (Neopreno)

Enveloppe Abs (Neoprene)

حزام البطن (نيوبرين)

REF J 06



UJM-J06-18-1

www.tynorindia.com

USER MANUAL

How To Use
Cómo Utilizar



कैसे उपयोग करें
Comment utiliser:
كيفية الاستخدام

1.



- The belt is wrapped around the waist.
- बेल्ट को कमर के आस पास बांधा जाता है ।
- El cinturón se enrolla alrededor de la cintura.
- La ceinture est enroulée autour de la taille.
- عم رصخلا لودم إزحلا فلأ مته

2.




- Keep it upright and in correct position as per directions of "up inside label".
- इसके लिए इसमें "अप इनसाइड लेबल" के निर्देशों का पालन किया जाना चाहिये ।
- Mantenerlo en posición vertical y en posición correcta según las instrucciones de "en el interior de la etiqueta".
- la garder et la position correcte selon les directives de l'étiquette intérieure.
- جعله متجها لأعلى وفي الموضع الصحيح كما تشير اتجاهات "الملصق الداخلي لأعلى"

How To Use
Cómo Utilizar



कैसे उपयोग करें
Comment utiliser:
كيفية الاستخدام

3.

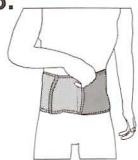

- Close the hook to a comfortable compression.
- इसके हुक को सही व सुविधाजनक संकुचन की स्थिति आने पर बंद कर दें ।
- Cerrar el gancho a una presión confortable.
- Fermez le crochet pour une compression confortable.
- قم بإغلاق المشبك إلى الحد الذي يمنحك الضغط المريح.

English

Abs Wrap (Neoprene)

INTENDED USE :

Abs Wrap (Neoprene) supports and compresses the abdominal muscles for waist line reduction and is used as a post operative or post delivery aid.

CONSTRUCTION:

Made out of three layered Neoprene foam bonded fabric, with a four way stretch, the top layer is made out of a special fabric with unbroken loops, which offers advantage of a flexible closing system for easy application and removal. The inner layer is made from strong nylon fabric which ensures a long life, good aesthetics and color fastness. Neoprene material provides four-way compression and retains body heat.

APPLICATIONS:

- To protect and bind surgical incisions of the abdomen
- Post operative abdominal care
- Reduce the discomfort of sudden rib cage expansion from deep breaths, cough or sneezing during recovery
- Used as a general tummy trimmer to reduce waist-line
- Post delivery, abdominal care & muscle toning
- In umbilical and ventral hernia

CAUTION/RECOMMENDATION :

- Remove while sleeping.

INSTRUCTIONS:

- Use this product on recommendation of a doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.