

Ankle Binder Urbane

Code- D 53

Size: S, M ,L, XL

Tynor Ankle Binder Urbane is a trendy and effective device to support, compress and partially immobilize the ankle following injury or sprain to control pain, oedema or inflammation. Composed of two components, compression sleeve and elasticized strap. The elasticized strap can be conveniently wrapped in a figure of 8 for strong and effective ankle support

Construction:

Knitted on the worlds most advance state of art CNC flat knit machine to provide , 3 D shaped fabric, in a double layer nylon yarn. Intermingled spandex provides the unique four way stretch to the fabric which gives a superior compression and ankle support. The strong spandex thread runs spirally in the matrix of nylon locks to ensure its free movement to allow uniform compression even on uneven diameters of the foot and ankle.

3d color pattern used gives the product sporty and highly pleasing looks

Applications:

- Sprains and strains of the ankle.
- Effective as a prophylaxis for sports person.
- Post operative and post cast care.
- Muscle, Tendon and ligament injuries and inflammation
- Oedema
- Contusion
- Arthritis

Features:

1. Elasticized figure of 8 strap improves ankle stability
2. Strong compression ensures improved performance
3. Easy pull-on application
4. Advanced 3D knit ensures a perfect anatomic shape and comfortable fitting.
5. Therapeutic warmth of the fabric ensures quicker healing.
6. Four way stretchable fabric for excellent fitting and comfort.
7. Good ventilation, Good sweat clearance
8. Soft fabric construction for enhanced comfort
9. Latex free and odorless
10. Excellent aesthetics with unique design patterns

Instructions:

- Product to be used for intended use only
- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - a. Impaired sensation
 - b. Impaired blood flow
 - c. Allergy rash or itching
 - d. If pain increases or persists consult your doctor in case of concurrent use with another medical device.
- Misuse, or improper use of the device can lead to adverse effects and decreased effectiveness
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.
- Dispose in a responsible manner as per hospital guidelines/Country's regulatory norms.

Recommendation: Hook Loop should be only attached on the elasticized strap and not on the compression sleeve fabric

Product Care:

Symbols

1. It is recommended to close hook loop closures before washing.
2. Hand wash fabric with mild detergent and water below 30°C without wringing.
3. Wipe the plastic parts with moist lint free cloth and mild detergent
4. Dry in shade on a flat surface,
5. Do not dry-clean.
6. Do not Iron.
7. Do not bleach

EC Rep:

MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Bullet Points:

- Latex free
- Sleek anatomic design
- Elegant 3D pattern
- Extremely comfortable

Contents: Nylon, Spandex

How to Use

1. Worn by a simple pull on application like socks with longer half upwards, so that heel fits in its position
2. Attach the elasticized strap around the ankle in a figure of 8