PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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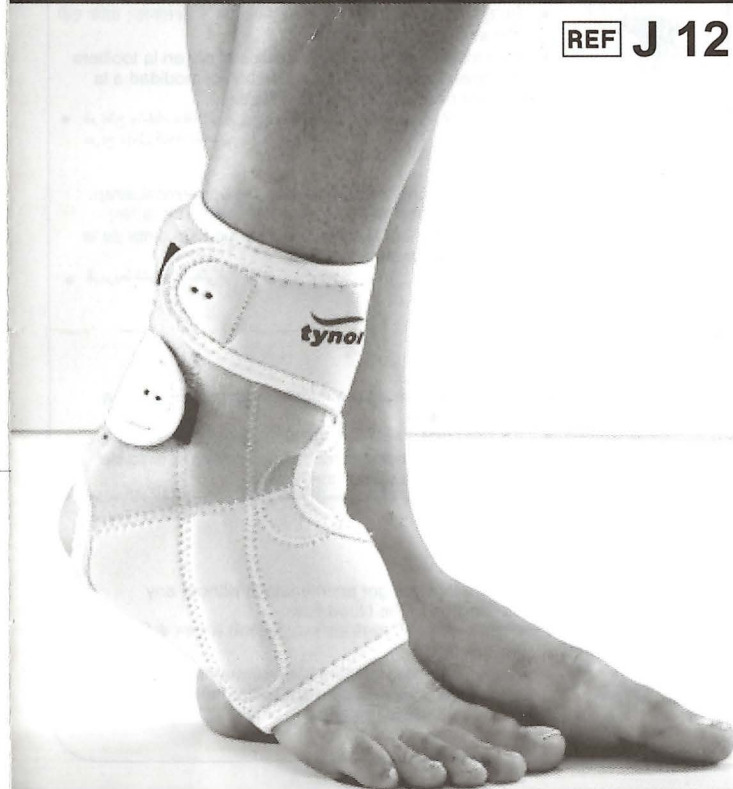
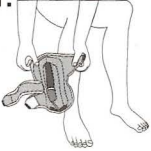
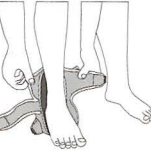
Ankle Support (Neoprene)

English

Ankle Support (Neoprene) is designed to give support, compression and controlled immobilization to weak or injured ankle. The stabilizing splint, supports the ankle where it is most needed, to prevent an ankle injury. It can be used in high impact, sporting activities or by individuals where repetitive ankle injuries are common. It also helps to control swelling, odema or sprains and rollovers of the ankle.

CONSTRUCTION:

Made out of three layered Neoprene bonded fabric, with a four way stretch, which ensures a snug fit and comfort of high degree. Anatomical, high tech design ensures perfect fitting. Semirigid PE splint immobilizes and supports the ankle where it most needed. The splinting action controls sprains and inversion and aversion rollovers. Anti slip silicon bands prevents slippage. Wrap around wearing design ensures easy application & removal.

APPLICATIONS:

- Chronic ankle instability
- Improves endurance
- Prophylaxis for sporting activity
- Prevention of repetitive ankle injuries
- Sprains and ankle roll overs
- Post operative and post cast care

CAUTION/RECOMMENDATION : Tighten straps optimally to ensure intimate fit, but should not impair the blood flow to the foot.

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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Tynor Orthotics Pvt. Ltd.