PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

Batch code

Manufacturing date

Manufacturer

Catalogue number

Manufactured by:

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, JLPL Ind. Area, Mohali - 160 055 (Pb) India
Consumer Care Cell: 1800-1212-656
E-mail: care@tynor.in or write to manufacturer
www.tynorindia.com

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English

INTENDED USE :

Ankle Support Urbane is aesthetically designed to give support and compression to weak or injured ankle. The stabilizing silicon insert match the ankle contours giving superior support with therapeutic compression. It can be used in high impact, sporting activities or by individuals where repetitive ankle injuries are common. It also helps to control swelling, odema or sprains and rollovers of the ankle.

CONSTRUCTION:

Knitted on an advance state of art CNC flat knit machine to provide , 3 D shaped fabric, in a double layer nylon yarn. Intermingled spandex provides the unique four way stretch to the fabric which gives a superior compression and ankle support. The strong spandex thread runs spirally in the matrix of nylon locks to ensure its free movement to allow uniform compression even on uneven diameters of the foot and ankle.

The silicon insert is anatomically designed with correct softness to support and anchor both the medial and lateral malleolus of the ankle.

3D color pattern used gives the product sporty and highly pleasing looks

APPLICATIONS:

- Ankle instability
- Prophylaxis for sporting activity
- Improves endurance in Gym and outdoor activities
- Prevention of repetitive ankle injuries
- Sprains and ankle roll overs
- Post operative and post cast care

INSTRUCTIONS^[1]

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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