English

#### POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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## PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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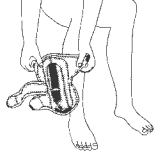
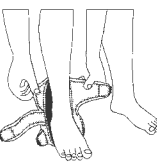
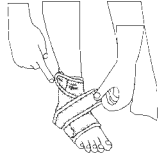
## Ankle Wrap (Neoprene)

### English

Tynor Ankle Wrap (Neoprene) is designed to give support, compression and partial immobilization to weak or injured ankle. High-performance design for active support to injured sprained, arthritic or inflamed ankles. Dual strapping allows controlled compression and custom fitting.

### CONSTRUCTION:

Made out of three layered Neoprene bonded fabric, with a four way stretch, which ensures a snug fit and comfort of high degree. Anatomical design ensures perfect fitting. Wrap around wearing design ensures easy application & removal

### APPLICATIONS:

- Chronic ankle instability
- Prophylaxis for sporting activity
- Prevention of repetitive ankle injuries
- Sprains and Strains
- Joint effusions and swellings
- Mild Arthritis and osteoarthritis of the ankle
- Post operative and post cast care
- Improves endurance

**RECOMMENDATION :** Tighten straps optimally to ensure intimate fit, but should not impair the blood flow to the foot.

### INSTRUCTIONS: [R]

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.