PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not iron.

Do not bleach.

Close hook loop fasteners before washing.

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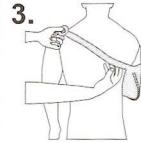
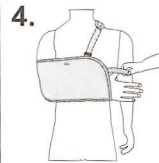
Pouch Arm Sling

INTENDED USE :

Pouch Arm Sling holds, supports and immobilizes the sprained, broken or surgically operated arm in the flexion position while it recuperates. Smart design to cover the cast and give extra protection to the injured arm.

CONSTRUCTION:

Made out of three-layered PU bonded fabric, each layer imparting to the fabric its unique properties.

- Outer layer with raising provides soft feel and plush looks
- Middle PUF layer imparts cushioning and wrinkle free property.
- Inner layer gives the required strength and non tear ability

The Oxy-pore model is however made from 3D perforated fabric for enhanced ventilation

Adjustable cotton sling allows proper fitting and positioning of the arm One touch opening and closing of the buckle allows easy application and removal of the pouch. Ethafoam shoulder pad takes the entire load of the arm and promotes patient comfort. Thumb cradle maintains the thumb in abduction, helps in relieving the fatigue to the thumb muscles.

APPLICATIONS :

- Fractures of humerus, radius, ulna, clavicles, scapula
- Clavicle bone fracture
- Post surgical or post cast care of the arm
- Sprained or a dislocated shoulder
- Any other condition requiring support or immobilization of the arm

RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS: [4]

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.