## PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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English

### INTENDED USE :

**Pouch Arm Sling (Urbane)** is designed for people who want to flaunt a sling like an accessory. It holds, supports and immobilizes the sprained, broken or surgically operated arm in flexion, while it recuperates. A chic, sleek and air ventilated design suits tropical conditions.

### CONSTRUCTION:

Pouch Arm Sling (Urbane) is made out of three-layered PU bonded fabric, each layer imparting to the fabric its unique properties.

- Outer layer with raising provides soft feel and plush looks
- Middle PUF layer imparts cushioning and wrinkle free property.
- Inner layer gives the required strength and non-tear ability

Design ensures that the chest remains uncovered. Highly ventilated system of interconnected holes further improves air circulation and patient comfort. Sling with double adjustment allows proper fitting and positioning of the arm.

P.U shoulder pad takes the entire load of the arm and provides comfort without any strain to the neck. Thumb cradle maintains the thumb in abduction and relieves thumb fatigue

### APPLICATIONS :

- Fractures of humerus, radius, ulna, clavicles, scapula
- Clavicle bone fracture
- Post-surgical or post cast care of the arm
- Sprained or a dislocated shoulder
- Any other condition requiring support or immobilization of the arm

### RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

### INSTRUCTIONS: [1]

- Discontinue use and seek guidance of a qualified doctor, in case of
  - Allergy rash or itching
  - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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