PRODUCT CARE

Do not wash.

Wipe with a clean, moist & lint free fabric to clean.

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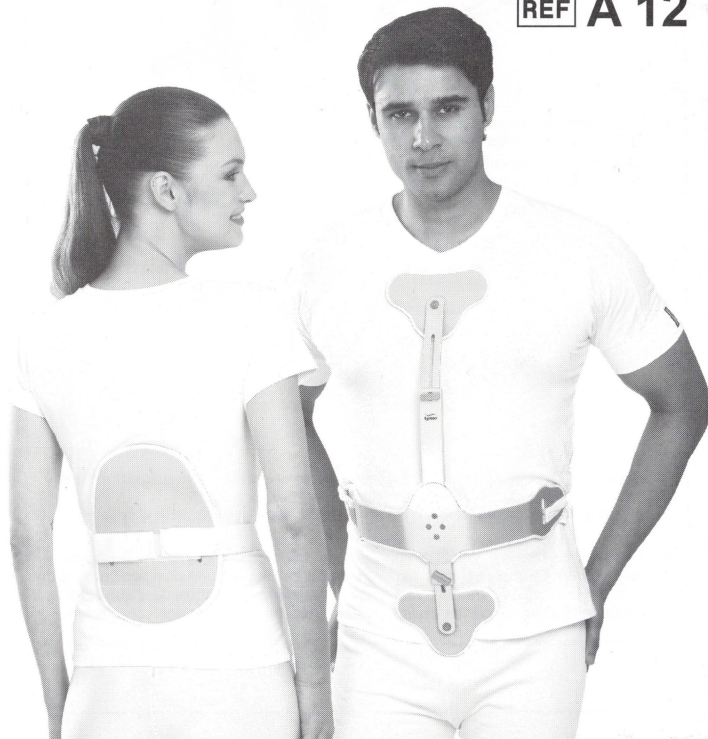
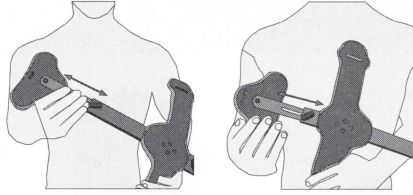
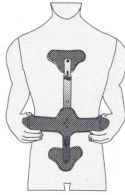
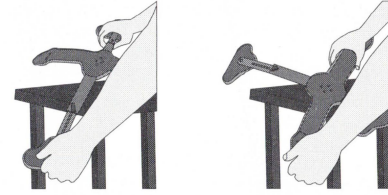
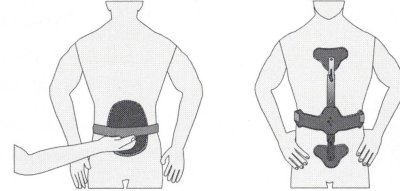
English

INTENDED USE :

Ash brace is designed to immobilize the vertebral column and maintain it in neutral or slightly hyper extension position. It provides stabilization of the spine encourages cervical and lumbar lordosis thus relieving pressure on the anterior parts of the vertebral bodies. Designed to act on three point pressure principle by exerting pressure against sternum and pubis on ventral and the lumbar region on the dorsal.

CONSTRUCTION:

Made out of aerodynamically shaped, telescoping, aluminum "O" section and malleable horizontal main body to form a cross bar. Knob and screw mechanism allows two way size adjustments of the brace on the vertical side and bending of the malleable horizontal main body on the horizontal side.

Anatomically contoured pubic and the sternal pads covered with ethafoam relieves pressure and ensure secure and comfortable fitting of the brace. A heavy duty strap attached to the two lateral sides of the brace with a broad rigid back pad help in the three point force application involving lumbar vertebra on one side and sternal and symphysis pubis on the other side.

Swivel sternal pad conforms to the chest and eliminates pressure on the chest.

APPLICATIONS:

- Pressure relief in osteoporotic bone collapse in the thoracic and lumbar spine
- Conservative management of vertebral bone tuberculosis
- To prevent deformity in ankylosing spondylitis
- Idiopathic hyperkyphosis of spine with chronic back pain or Scheuermann's disease
- Conservative treatment of in-operable vertebral fractures in the lumbar and thoracic spine
- Diskites, inter vertebral disc prolapse or herniation
- Un-displaced stable fractures and compression fractures of the lower thoracic and upper lumbar vertebral column
- Temporary, postoperative spine immobilization & stabilization

CAUTION/RECOMMENDATION : Bend the horizontal bar to shape for good fitting and comfort.

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use of another medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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