PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing

Wipe with moist lint free cloth

Dry in shade in the insert provided in box

Apply talcum powder after every wash

Do not place on heater or use a hair dryer

Avoid dropping on hard surface

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

- LOT** Batch code
M Manufacturing date
M Manufacturer
REF Catalogue number

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English

INTENDED USE :

Tynor Breast Prosthesis is an artificial breast that gives the appearance, feel, touch, weight, softness, of a real breast. It looks realistic and feel more natural as compared to other available products. Counters the weight of other breast, helps in balancing the two sides & keeps the shoulder even and straight the posture. Specially designed bra's with pocket are used to hold the breast in its position

Made from high quality silicone material similar to our body tissue. Silicone is skin friendly and highly inert material

PRODUCT CARE:

- Keep the product away from sharp edges.
- Misuse, or improper use of the product can lead to decreased effectiveness
- Follow instructions mentioned on the instructions manual carefully.
- Avoid excessive pressure on product.
- Don't puncture or rupture the product
- It is recommended to remove the product while sleeping.
- Avoid excess pressure on product.
- Wipe the product with moist lint free cloth.
- After use, Do not place the product on a flat surface. Place the product in the box only.
- Wash regularly with plain water at room temp.
- Discontinue use and seek guidance of a qualified doctor, in case of, allergy rash or itching

APPLICATIONS:

- Restoring the original anatomy
- Restoring self-esteem
- Prevent scoliosis, post mastectomy
- Low back and cervical muscle strain

INSTRUCTIONS:

- Silicone breast prosthesis can be used only after minimum of eight weeks of therapy.

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