PRODUCT CARE

Misuse, or improper use of the product can lead to decreased effectiveness.

Keep away from sharp edges

Follow instructions mentioned on the instructions manual carefully.

Wipe the plastic cover with moist lint free cloth and mild detergent

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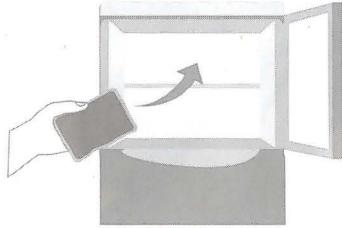
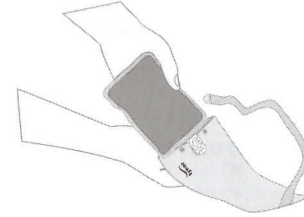
English

INTRODUCTION:

Cool Pack is a convenient device to provide cold compress. Cold compress helps in reduction of inflammation, which protects by slowing the metabolic rate around the injured tissue, reduce oedema and bleeding. Cold compress lowers fever, can be used after an acute injury or surgical procedure. It can also be used to treat ocular allergies, ocular itching and to reduce swelling around the eyes.

CONSTRUCTION:

The gel pouch is made out of soft and flexible PU/PVC film, filled with a hydrophilic gel. Fabric cover prevents cryo burns and ensures convenience.

APPLICATIONS:

Cold compress

- High fever
- Oedema
- Bleeding
- Inflammation and soft tissue injuries
- Regain working of an overworked muscle
- Post surgical care
- Management of strains, sprains, bruises

CAUTION:

- Cool pack application should be avoided by patients with certain medical conditions, such as for patients who have rheumatoid arthritis, Raynaud's Syndrome, cold allergic conditions, paralysis, or areas of impaired sensation.
- Individual with problems in sensing cold should use extreme caution with this product.
- When using for treatment of post-surgical trauma, follow instructions of the Doctor.
- If contents are exposed to the skin or eyes, rinse with water immediately. If irritation or redness persists, consult a physician.
- Do not use if gel pack is punctured or damaged.
- Do not sit or lie on gel pouch, excessive weight above 25Kg, may cause pouch to rupture or leak.

INSTRUCTIONS:

- Always use the Cool pack with Cloth Cover
- Limit the ice application to no more than twenty minutes at a time
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Keep out of reach of children.

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