

PRODUCT CARE



Wipe with a clean, moist & lint free cloth to clean



Air dry properly, before wearing and after cleaning



Tynor Orthotics Pvt. Ltd.

D-111, Industrial Area Phase-7, Mohali- 160 055, (Punjab) India

e-mail: marketing@tynorindia.com • www.tynorindia.com

Customer Care : 0091-172-4388111


ISO 9001 | CRISIL | W.H.O. G.M.P. | ISO 13485 | CE | US FDA registered

www.tynorindia.com

tynor

Cast Cover
कास्ट कवर
Funda De Enyesado
جيرة كاست كوفر

REF C 16/C 19

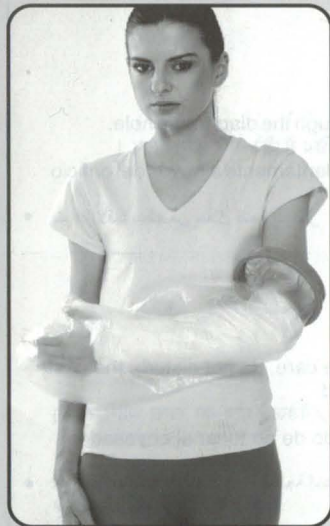

Cast Cover (Arm)



Cast Cover (Leg)

UM-C16/C19-14-1

USER MANUAL

How To Use Cómo Utilizar



कैसे उपयोग करें كيفية الاستخدام

1.



- Hold the plastic ring perpendicular to the arm/leg.
- प्लास्टिक रिंग को बाजू/पैरों पर लंबे रूप में रखिए।
- Sostiene el anillo de plástico perpendicular al brazo / pierna.
- عقد حلقة بلاستيكية عمودي على الذراع / والساق

2.



- Slide slowly through the diaphragm hole.
- सावधानीपूर्वक पटल छिद्र में धीरे-धीरे खिसकाइए।
- Deslice la funda lentamente a través del orificio del diafragma.
- يتم انزلاقها ببطء من خلال فتحة الحاجز

3.



- While taking due care, do not disturb, the cast/ bandage /wound.
- खिसकाते हुए कास्ट/बैंडेज/घाव का खास ध्यान रखें।
- Teniendo cuidado de no tocar el enyesado/ vendaje/herida.
- مع توفير العناية اللازمة, دون الحاجة إحداث اضطراب للجبيرة / الضمامة / الجرح

Note : Please refer to the "How to use" instructions shown on the packaging.

Cast Cover

English

Tynor Cast Cover is a generic cast/bandage/wound protector, which makes the fitting water tight and keeps your cast/bandage/wound dry during the shower or otherwise. It is reusable.

All casts POP or synthetic and the cast padding underneath needs to be kept dry.

CONSTRUCTION:

Tynor Cast cover is made from a durable, transparent PVC fabric. It covers the limb when applied and the sealing is done with an expandable hole drilled into a latex diaphragm which is held in place with a rigid plastic ring.

APPLICATIONS:

- Used as a generic cast/bandage/wound protector, during shower or bathing.
- Can be used as a cast/bandage/wound protector, during transportation in rain/wet weather conditions.



CAUTION/RECOMMENDATION/INSTRUCTIONS :

- For therapeutic care, during shower/bathing only.
- When submersing, do not put weight on plastic ring.
- It is not recommended to be used during recreational water activities.
- With children, adult supervision recommended.
- Be extra careful in wet surroundings, It is slippery when wet.
- Consult your doctor if the diaphragm causes discomfort or swelling.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.