

PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.



Do not Iron.



Do not bleach.



Close hook loop fasteners before washing.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :



Batch code



Manufacturing date



Manufacturer



Catalogue number



Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001



W.H.O.
G.M.P.

ISO
13485



US FDA
registered

UM-B03-17-1



Cervical Collar Hard Adjustable

सर्वाइकल कॉलर हार्ड एडजस्टेबल
Collarin Cervical Firme Ajustable
الطوق العنقي الثابت القابل للضبط

REF B 03



www.tynorindia.com


USER MANUAL

How To Use Cómo Utilizar




कैसे उपयोग करें كيفية الاستخدام

1.

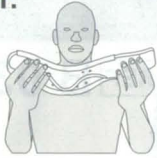

- Measure the neck height, look for the right size of the collar.
- गर्दन की ऊंचाई मापें, उसके अनुसार सही आकार का कॉलर तलाशें।
- Mida la altura del cuello y determine el tamaño adecuado del collarín.
- قم بقياس طول الرقبة ثم ابحث عن المقاس المناسب للطوق.

2.

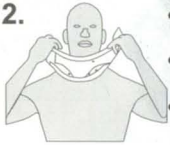


- Widen the two parts of collar to the exact measurement of the neck for a customized size.
- कॉलर के दोनों हिस्सों को चौड़ा कर लें ताकि वह गर्दन के सटीक माप के अनुसार हो जाए।
- Extienda las dos partes del collarín hasta la medida exacta del cuello para obtener un tamaño personalizado.
- ثم قم بتوسيع طرفي الطوق لتناسب مقياس الرقبة بشكل دقيق بهدف الوصول إلى الحجم المخصص.

3.



- Wrap it around the neck snugly with the trough of the collar matching the chin.
- कॉलर का सिरा ठोड़ी के सीधे में रखते हुए आराम से गर्दन पर लपेट लें।
- Colóquelo en torno al cuello cómodamente con el corte anatómico a la altura de la barbilla.
- قم بلف الطوق حول الرقبة بشكل مريح مع مطابقة حوض الطوق للذقن.

4.

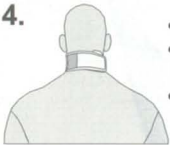

- Fasten hook to the loop tape to a comfortable fit.
- हुक को लूप टेप पर इस तरह कमें कि यह आरामदायक स्थिति में फिट हो जाए।
- Ajuste la hebilla hasta el punto en que lo sienta cómodo.
- اربط المشبك بشرط الحلقة للحصول على إحكام تام.

Cervical Collar Hard Adjustable

English

Cervical Collar Hard (Adjustable) used for supporting, immobilizing or adjusting the neck in the flexion, extension, or hyperextension position. Recommended where a rigid immobilization of the cervical region is required.

CONSTRUCTION:

Cervical Collar Hard (Adjustable) is made out of polyethylene sheet, the edges are padded with molded Ethafoam round tube and covered with vinyl fabric of a superior GSM and quality. Ethafoam has the distinct advantage of a very high cushioning coefficient for improved patient comfort, is very light in weight, is hypoallergenic, has good shearing strength apart from being odorless and non absorbent. Air ventilation holes ensure proper air circulation, thereby improved patient comfort and compliance. Hook loop closure ensure close fitting of the collar around neck, its quick, easy application and removal.

The height of the collar can be varied for accurate dimensional adjustments to suit the individual neck heights.

APPLICATIONS:

Recommended where a rigid immobilization of the cervical region is required

- Cervical Spondylitis
- Cervical disc disease
- Cervical neuralgia
- Fractures of cervical spine
- Torticollis
- Early Rheumatoid disease
- Treatment of whip lash injuries
- Other ortho/neuro conditions where rigid immobilization of the neck region is required.



CAUTION/RECOMMENDATION : Height adjustment in collar should only be used for fine matching of the collar height to the neck height and not for sizing purpose.

INSTRUCTIONS

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.