Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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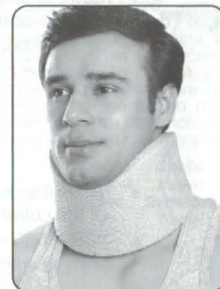
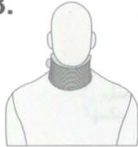
Cervical Collars

English

Cervical Collars are used for supporting, immobilizing or adjusting the neck in the required position, generally extension position. Different variants are designed for varying degrees of immobilization as required for different conditions.

CONSTRUCTION:

Soft collars come in three variants

A. Soft B. Soft (firm density) C. Soft with support.

They are produced from soft PUF or High density PUF, and can also be reinforced by LDPE sheet as in C. Collar Soft with Support. The chief purpose is to provide support, optimal immobilization and high comfort to the neck. It is covered with soft, hypoallergenic cotton stockinet. The body of the collar is perforated to provide ventilation for patient's comfort. Hook loop closure ensured close fitting of the collar around neck, its quick, easy application and removal.

APPLICATIONS:

	Cervical Collar Soft	Collar Soft Firm Density	C.Collar Soft with Support
• Early Rheumatoid disease	✓	✓	
• Whip lash injuries	✓	✓	
• Geriatric Care, General Prophylaxis	✓	✓	
• Neck sprain, stiff neck	✓	✓	✓
• Cervical Spondylitis	✓	✓	✓
• Cervical disc disease		✓	✓
• Torticolles		✓	✓
• Post operative care		✓	✓

INSTRUCTIONS: [R]

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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