PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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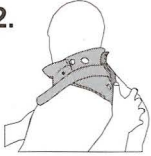
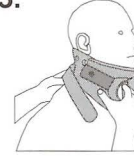
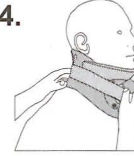
Cervical Orthosis (Philadelphia)

English

An advanced collar with scientific design which ensures complete immobilization on one hand and perfect comfort on the other. Frontal opening allows it to be used in patients with tracheotomy, It also helps in good ventilation of the area. Swollen balloon shape makes it an ideal collar for treating burn patients.

CONSTRUCTION:

Moulded out of linear cross linked polyethylene foam (Ethafom or Plastozote) which has a very dense cell structure which imparts strength and rigidity to the soft cushiony matrix. It is highly inert, odorless, non absorbent and hypoallergenic in nature.

The frontal and the rear splints are anatomically designed to support the neck, limit the flexion and extension movement of cervical spine and provide a very effective immobilization of the area.

Pre moulded chin and the balloon design gives only a rim contact to the body without involving the neck surface, which ensures rigid immobilization on one hand and complete comfort to the other. Two piece designs with an inbuilt chin rest also helps in better immobilization.

APPLICATIONS:

- Cervical spondylitis
- Cervical spondylolisthesis
- Post operative care of the cervical area
- Stable cervical vertebral fractures.
- Post operative management after tuberculous surgery
- Burns care of the cervical region
- Extrication and transportation of the accident victims
- Post traction stabilization
- All other ortho and neuro conditions, requiring firm immobilization of the cervical region.

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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