PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

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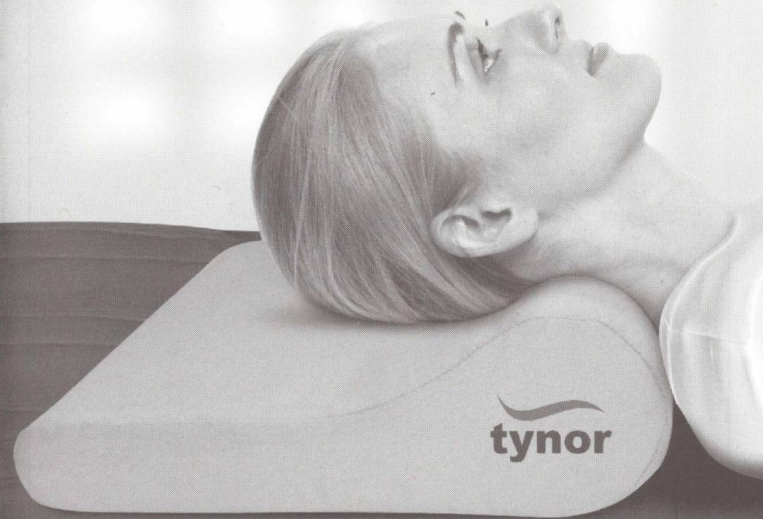
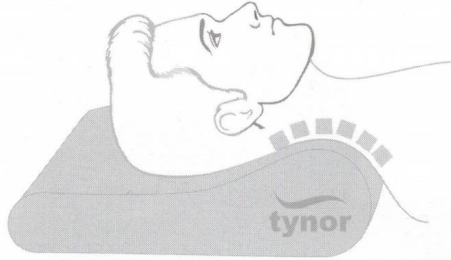
English

INTENDED USE:

Cervical pillow is designed to keep the neck in the slightly hyperextension position at night to counter the stress and strain of the day where we tend to keep the neck in the opposite "flexion" position. It prevents and helps cure the cervical problems while you sleep by promoting the healthy, neutral posture of the cervical spine.

CONSTRUCTION:

It is made out of high quality and high density PU foam to provide a firm but a comfortable and soft support to the neck and head. The PU foam is anatomically shaped to provide a slight hyper extension of the neck. It is covered with a soft velvet fabric.

APPLICATIONS:

- Used for patients suffering from cervical spondylosis or back pain
- To correct snoring induced sleep apnea
- As a prophylactic aid against cervical problems
- Can also be used on surgical tables, recovery rooms and treatment tables

RECOMMENDATION :

- Do not sleep on stomach if you have neck pain or cervical problems.
- The use of additional removable cover is recommend to keep the pillow neat & clean.

INSTRUCTIONS:

1. Use this product under strict guidance of a qualified doctor.
2. Keep it under neck while asleep
3. Adjust and position the pillow so as to give the neck slight hyper extension position and comfort.
4. Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Pain increases or persists
5. Consult your doctor in case of concurrent use with any other medical device.
6. Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
7. Follow instructions mentioned on the box carefully.

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