

## PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.



Do not Iron.



Do not bleach.



Close hook loop fasteners before washing.

EC REP

MT Promedt Consulting GmbH  
Altenhofstrasse 80  
D-66386 St. Ingbert  
Germany  
tel.: +49 (0) 6894 581020  
fax: +49 (0) 6894 581021  
e-mail: info@mt-procons.com  
www.mt-procons.com

### Symbol Explanation :



Batch code



Manufacturing date



Manufacturer



Catalogue number



### Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JIPL, Mohali-160 055, (Pb) India  
e-mail: marketing@tynorindia.com • www.tynorindia.com  
Customer Care : 0091-172-5288111

ISO 9001 | CRISIL | W.H.O. G.M.P. | ISO 13485 | CE | US FDA registered

www.tynorindia.com



**Chest Binder**  
चेस्ट बाइंडर  
Faja Pectoral  
Cartilage Thoracique  
المحزم الصدري

REF A 11



UM-A11-1B-1

USER MANUAL


**How To Use**  
**Cómo Utilizar**



**कैसे उपयोग करें**  
**Comment utiliser**  
**كيفية الاستخدام**



- The chest pad is placed over the sternum.
- चेस्ट पैड को उरोस्थि (sternum) पर लगाया जाता है।
- La almohadilla pectoral se coloca sobre el esternón.
- Le coussin thoracique est placé sur le sternum.
- يتم وضع الوسادة الصدرية على عظم القص.

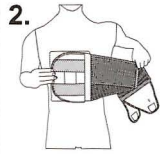

- The belt is wrapped around the waist .
- बेल्ट को कमर के चारों तरफ कसा जाता है।
- La tira se ata alrededor de la cintura manteniéndola recta y en la posición correcta.
- La ceinture est enroulée autour de la taille
- ثم لف الحزام حول الخصر مع إبقائه مستقيماً وفي وضعية صحيحة

**How To Use**  
**Cómo Utilizar**



**कैसे उपयोग करें**  
**Comment utiliser**  
**كيفية الاستخدام**



- The strip is attached as instructed in the "up inside label."
- "अंदर के लेबल" ("up inside label") में दिए गए निर्देशों के अनुसार इसे ऊपर की ओर सीधा और सही अवस्था में रखें।
- La tira se ata según las instrucciones de la "etiqueta del interior".
- En la maintenant droite et en position correcte selon les directives à « l'intérieur de l'étiquette ».
- يتم إرفاق شريط وفقاً للتعليمات في "حتى داخل التسمية"



- Apply hook tape to the loop tape to a comfortable compression.
- सुविधाजनक दबाव के लिए लूप टेप में हुक टेप लगाएं।
- Aplique la cinta de velcro al cierre de velcro para obtener una compresión confortable.
- Appliquer du ruban à crochets à la bande de la boucle à une compression confortable.
- جیرمٹغضبدرعشتی تد (بجول) ی قلحلا طیرشلا عم (لغو) طبار لا طیرشلا تحبثو

## How To Use Cómo Utilizar



## कैसे उपयोग करें Comment utiliser كيفية الاستخدام

3.



- The strip is attached as instructed in the "up inside label."
- "अंदर के लेबल" ("up inside label") में दिए गए निर्देशों के अनुसार इसे ऊपर की ओर सीधा और सही अवस्था में रखें।
- La tira se ata según las instrucciones de la "etiqueta del interior".
- En la maintenant droite et en position correcte selon les directives à « l'intérieur de l'étiquette ».
- يتم إرفاق شريط وفقا للتعليمات في "حتى داخل التسمية".

4.



- Apply hook tape to the loop tape to a comfortable compression.
- सुविधाजनक दबाव के लिए लूप टेप में हुक टेप लगाएं।
- Applique la cinto de velcro al cierre de velcro para obtener una compresión confortable.
- Appliquer du ruban à crochets à la bande de la boucle à une compression confortable.
- جیرم طغضیدر عشتی تم (بولول) ی قحلا طیرشلا یم (لموهر) طبارنا طیرشلا تبثو

## Chest Binder

### English

#### INTENDED USE :

**Chest Binder** is applied to the thoracic region to compress and bind the rib cage and provide splinting to the sternum while allowing sufficient flexibility for comfortable breathing.

#### CONSTRUCTION:

Made out of superior quality elastic webbing. It contains heat resistant rubber with high modulus of elasticity which ensures a long life of the binder. Broad panels of Hook Loop closure ensure strong and snug fitting of the binder around the chest, its quick easy application and removal.

The chest pad is made out of soft PU and etha foam with high cushioning coefficient to provide effective binding of the rib cage, vibration free splinting and immobilization of the sternum.

#### APPLICATIONS:

- Stabilization of fractures of sternum
- Post cardio thoracic surgery (Reduce the discomfort of sudden rib cage expansion from deep breaths, cough or sneezing during recovery).
- To protect and bind surgical incisions of chest
- Ease the process of expulsion of phlegm out of lungs after surgery which might otherwise lead to infectious lung pleurisy and other serious complications.



**CAUTION/RECOMMENDATION** : Center of the Chest pad be located in the furrow of the chest. Compression be carefully controlled to give optimal immobilization and pain relief.

#### INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.