## PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not iron.

Do not bleach.

EC REP

MT Promed Consulting GmbH  
 Altenhofstrasse 80  
 D-66386 St. Ingbert  
 Germany  
 tel.: +49 (0) 6894 581020  
 fax: +49 (0) 6894 581021  
 e-mail: info@mt-procons.com  
 www.mt-procons.com

### Symbol Explanation :

- LOT** Batch code
- Manufacturing date
- Manufacturer
- REF** Catalogue number

**Tynor Orthotics Pvt. Ltd.**

169-170, Sec 82, Ind. Area, JLPL, Mohali- 160 055, (Pb) India  
 e-mail: marketing@tynorindia.com • www.tynorindia.com  
 Customer Care : +91-172-5288111

ISO 9001 | CRISIL | W.H.O. G.M.P. | ISO 13485 | CE | US FDA registered

www.tynorindia.com

## Compression Garment Arm

कम्प्रेसन गारमेंट आर्म  
 Prenda Brazo De Compresión  
 Vetement De Compression Bras  
 كم ملابس ضاغطة


**REF | 74, 75 & 77**

Compression Garment Arm Sleeve

Compression Garment Arm Sleeve + Mitten (with Thumb)

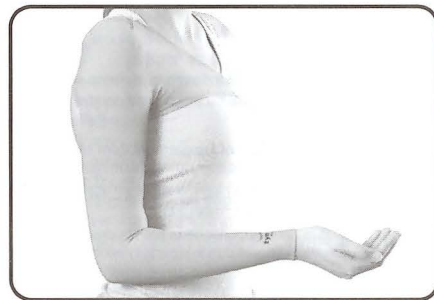Compression Garment Arm Sleeve with Shoulder Cover

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English

### INTENDED USE :

**Compression Garments** are specialized clothing devices, that provide support and graduated compression, in Burns, Lymphedema and other vascular conditions. Compression garments include sleeves, gloves, tights, and a variety of specially fitted garments for individual needs.

### CONSTRUCTION:

Stitched from high quality, Rayon and Spandex yarn fabric, which is woven on state of art weaving machines. This special fabric is aerated, light in weight and has a strong four way stretching and compression. The final garment is designed to get an anatomical shape with a graduated and a controlled compression.

### APPLICATIONS :

- Burns
- Chronic peripheral venous insufficiency
- Varicose veins
- Spider Veins (mild varicosities)
- Deep Vein Thrombosis (DVT)
- Lymphedema
- Phlebitis
- Economy Class Syndrome (ECS)
- Lipodermatosclerosis
- Blood clots, post surgery or injury.
- Pregnancy

### CONTRAINDICATIONS:

Patients with peripheral obstructive arterial disease, peripheral neuropathy, congestive heart failure, septic phlebitis or oozing dermatitis should not wear or consult the vascular physician for expert advice.

### RECOMMENDATION :

- If you use lotion on your arms, let it dry before you put on the garments.
- Use a little baby powder or corn starch on your arms. This helps the garment slide up.
- Put on rubber dish washing gloves on your hands to adjust and smoothen the garment.

### INSTRUCTIONS: [1]

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
  - a. Impaired sensation.
  - b. Impaired blood flow.
  - c. Allergy rash or itching.
  - d. Pain increases or persists.
- Consult your doctor in case of concurrent use with another medical device.
- Excessive physical exertion can lead to a risk of rise in blood pressure. Take extra precaution.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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