

PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.



Do not Iron.



Do not bleach.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :



Batch code



Manufacturing date



Manufacturer



Catalogue number

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPI, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001



W.H.O.
G.M.P.


ISO
13485



US FDA
registered

Elbow Support
एल्बो सपोर्ट
Codera Elástica
Support De Coude
دعامة الكوع

REF E 11

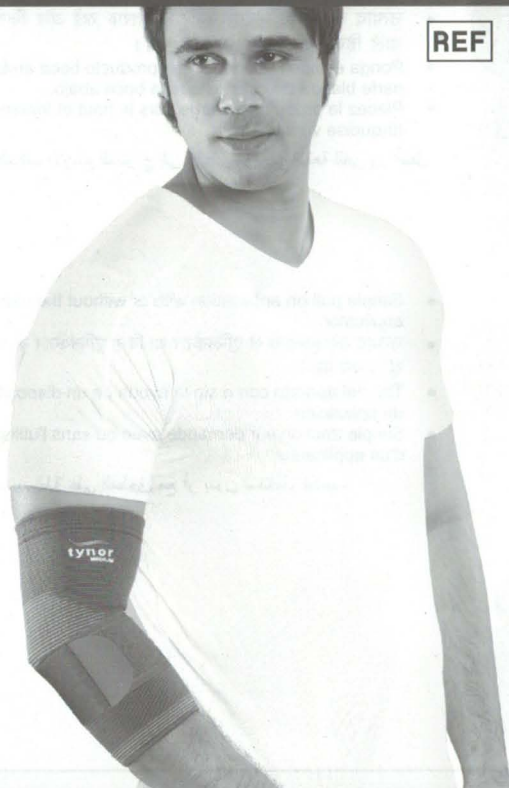

UM-E11-18-1

www.tynorindia.com

USER MANUAL

Elbow Support

English

Elbow support is widely used in orthopaedic practice to provide firm compression, warmth & support to the forearm and the elbow joint, to allay pain inflammation and stiffness, generally associated with old age, arthritis, sports injury etc.

CONSTRUCTION:

Woven on the state of art CNC looms to provide seam less, tapered tubes, in a double layer of nylon fiber, Spandex and long staple cotton yarn. Nylon improves life and suppleness of the product. Nylon is aesthetically more pleasing and offers color fastness. Cotton is dermophillic, hypoallergenic, warm in winters, cool in summers. Interwoven rubber thread, has a high modulus of elasticity and heat resistance. Intermingled spandex provides the unique four way stretch to the fabric.

The computer controlled knitting offers a unique advantage of soft elbow and fine grip to the edges. It also enhances designing and aesthetics of the product.

APPLICATIONS:

- Helps reduce inflammation and relieves stiffness
- Gives firm compression and therapeutic warmth to inflamed elbow.
- Tennis elbow, Golfers elbow
- Tendonitis
- Offers protection against further injury.
- Psychological support to geriatric patients.
- For arthritic elbow conditions.
- Sports injury, sprains and twists.
- Post-surgical rehabilitation
- Burn care

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

How To Use
Cómo Utilizar



कैसे उपयोग करें
Comment utiliser:
كيفية الاستخدام


1.



- Position the product wider side up and turquoise soft spot facing down.
- उत्पाद का चौड़ा हिस्सा ऊपर की तरफ रखें और फिरोजी वाले हिस्से को निचे की तरफ रखें।
- Ponga el lado más ancho del producto boca arriba y la parte blanda de color turquesa boca abajo.
- Placez le produit plus large vers le haut et fontanelle turquoise vers le bas.

• ضع الجانب الأوسع للمنتج إلى الأعلى وواجهة البقعة للغير وز أسفل

2.

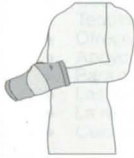

- Simple pull on application with or without the use of an applicator.
- उत्पाद को आसानी से एप्लिकेटर या बिना एप्लिकेटर के प्रयोग खींच कर पहनें।
- Tire del aparato con o sin la ayuda de un dispositivo de aplicación.
- Simple traction sur demande avec ou sans l'utilisation d'un applicateur.

• إسحب ببساطة على التطبيق مع أو بدون استخدام قضييب

How To Use
Cómo Utilizar



कैसे उपयोग करें
Comment utiliser:
كيفية الاستخدام

3.



- Smoothen wrinkles and position the turquoise soft spot exactly in the centre of the elbow.
- उत्पाद को पहनने के बाद असमता को ठीक करें और फिराजी रंग वाले हिस्से को कोहनी के मध्य में लाए।
- Alise las arrugas y ponga la parte blanda de color turquesa exactamente en el centro del codo.
- Lisser les plis et positionner la fontanelle turquoise exactement au centre du coude.

• قم بتسيب التجاعيد و وضع البقعة اللينة للغير وز بالضبط في وسط الكوع