PRODUCT CARE

Wipe with a clean, moist, lint free fabric to clean.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

- LOT** Batch code
M Manufacturing date
M Manufacturer
REF Catalogue number

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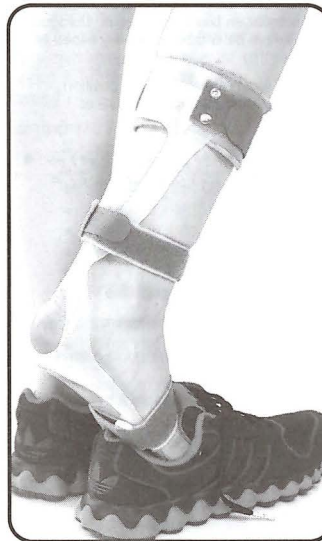
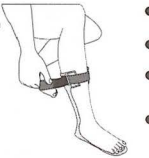
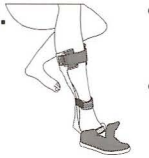
English

INTENDED USE :

Foot Drop Splint is designed with a dorsal shell made of polypropylene with a leaf spring action to arrest the plantar flexion movement and lift the foot. It supports and stabilizes the ankle and foot in all foot drop conditions. Designed to aid in swing phase toe clearance and to reduce plantar flexion impact in the early stance.

CONSTRUCTION:

Highly anatomic and perforated design. Made out of durable PP polymer. Its thin walled construction has been optimized for increased resilience and designed to be worn with the shoe. Unique leafspring action adds bounce and aids in toe clearance while walking.

Air filled bubble sheet liner ensure soft, comfortable interface with the PP shell. Hook loop bands with reverse buckle mechanism ensure strong grip of the product to the foot.

APPLICATIONS:

- Prevention and correction of foot drop
- Neurological deficit
- Peripheral nerve paralysis
- Peroneal/Peritibial Nerve or Muscle damage
- Ankle or Plantar flexion contracture
- Functional Alignment and stabilization of the foot and ankle
- Post operative care
- Burn patients

CAUTION/RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.
- The contours can be customized using hot air gun to improve effectiveness and comfort.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.