

PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.



Do not Iron.



Do not bleach.



Close hook loop fasteners before washing.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :



Batch code



Manufacturing date



Manufacturer



Catalogue number

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001



W.H.O.
G.M.P.

ISO
13485



US FDA
registered

www.tynorindia.com

Forearm Splint

फोरआर्म स्प्लिन्ट

Tablilla De Antebrazo

جبيرة الذراع

REF E 30



UM-E30-17-1

USER MANUAL

How To Use Cómo Utilizar



कैसे उपयोग करें كيفية الاستخدام

1.



- Aluminum strips are pushed out one by one. The contours of the splints are matched to the contours of the wrist and the degree of Dorsi flexion required. The customized splints are then pushed back into the pockets.
- एल्युमिनियम की कमटियों को एक के बाद एक निका लें। कमटियों का आकार कलाई के आकार से और पीछे की ओर मुड़े कोण के साथ मिलान करें। इस प्रकार से बनाई गई कमटियों वापिस पॉकेट्स में डाल दें।
- Las tiras de aluminio se presionan una a una. El contorno de la tablilla se ajusta al contorno de la muñeca y al grado de dorsiflexión requerido. Las tablillas adaptadas a la forma del se insertan luego en los bolsillos.

• يتم دفع شرائح الألومنيوم واحدة تلو الأخرى. ويلانم محيط الدعائم مع محيط المعصم ودرجة الانثناء الظهراني المطلوبة. ثم يتم إدخال الدعائم المعدلة مرة أخرى في الجيوب.

2.




- The injured hand is pushed into the tubular opening of the product.
- चोट लगे हुए हाथ को नलिका के समान उत्पाद के प्रारंभिक छिद्र में डालें।
- La mano lesionada se introduce en la apertura tubular del producto.
- وتوضع اليد المصابة في فتحة أنبوبية من المنتج.

3.



- The proximal edge of the product is aligned to the palmar crease.
- उत्पाद का बाहरी किनारा पंजे की चौड़ाई के अनुरूप सैट कर लें।
- El borde proximal del producto se alinea al pliegue palmar.
- ويتم موازنة الحافة الدانية للمنتج مع خط راحة اليد. ثم تربط.

4.

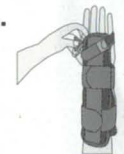

- The Velcro rejoiners are fastened to a comfortable compression one by one starting from the wrist end.
- वेलक्रो की मदद से सुविधाजनक संकुचन के लिए इन्हें शुरु से आखिर तक एक-एक कर के बाँधें।
- Los cierres de velcro se tensan uno a uno para que ejerzan una compresión confortable, comenzando por el del lado de la muñeca.
- فيلکرو بضغط مريح واحدة تلو الأخرى بدءاً من نهاية المعصم.

Forearm Splint

English

Forearm Splint is a long brace designed to provide splinting support to radius and ulna and immobilize, firmly the forearm in various orthopedic conditions. It maintains the wrist in the functional position.

CONSTRUCTION:

Made from strong three layered bonded fabric with the middle layer made of open celled foam to ensure good cushioning. Rigid support is provided on both dorsal and ventral side of the forearm by pre contoured, malleable aluminum stays, which can be molded for a customized fit. Velcro closures provide snug fitting, easy application and removal of the splint.

Forearm Splint is designed to maintain wrist in slight dorsi flexion while allowing full range of motion to fingers and thumb. The splint can be removed to permit, therapy sessions, washing and hygiene. Designed to fit right as well as left hand.

APPLICATIONS:

- Sequel to a fracture. Post cast management
- Post surgical protection and rehabilitation
- After extensor tendon repair of the wrist
- Carpal tunnel syndrome
- Sprains and strains
- All conditions requiring orthopedic support to forearm

▲ **CAUTION/RECOMMENDATION** : Remove aluminum splints, customize and reinsert before wearing.

INSTRUCTIONS: [1]

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.