PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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English

INTENDED USE :

Functional knee Support, offers the advantage of controlled compression around the knee and a rigid lateral support and immobilization. It allows normal flexion and free movement of the knee joint. Because of front opening is convenient to apply and remove.

CONSTRUCTION:

Made out of strong elasticized fabric. It is shaped anatomically to provide uniform compression around the knee without bunching during knee bending. Strong hinged splints provide lateral stability and support, but allow normal flexion of the knee joint. Anterior opening design allows easy application, particularly in geriatric patients or when the surrounding area is injured or when the knee is swollen or disfigured. Hook loop closures provide controlled compression and close contoured fitting apart from easy application and removal.

APPLICATIONS:

- Mild to moderate instability of the knee and support to weak knees
- Mild to moderate OA
- Control painful knee movements in arthritis
- Patello femoral pathology
- Patellar fracture
- Lateral patellar subluxation
- Lateral patellofemoral malalignment
- Miniscus injury
- Tendon or ligament injury of the knee
- Muscular sprains and strains of the area
- Post operative care of the knee
- Prophylaxis in sports activities
- Advanced Geriatric patient care

CAUTION/RECOMMENDATION :

- Use this product on recommendation of a doctor.
- Match the axis of the hinge of the splint and knee at the time of application.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use of the device with excessive physical exertion, it can lead to restriction of performance of heart and lungs (risk of rise in blood pressure).
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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