PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Do not wring the product

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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English

INTENDED USE :

Tynor Hernia Belt offers a gentle and constant relief from reducible inguinal hernia by providing adjustable and comfortable pressure to the hernia. On wearing the belt the occurrence of herniation gets eliminated. Tynor Hernia Belt is available in skin color, can be worn inconspicuously under clothing.

CONSTRUCTION:

Tynor Hernia Belt is made of out of soft and skin friendly materials. The moulded and removable foam hernia pads are designed to provide gradual pressure around the affected area with focused pressure on the hernia, which pushes the inguinal hernia back. Velcro closures offer flexibility in both waist and perineal (leg) straps. They can be easily adjusted for a customized fitting and to achieve the desired pressure of the pads. Pads can be removed to treat a single hernia.

APPLICATIONS:

- Provide a constant, and comfortable pressure to reduce the hernia by gently pushing it back.
- Inguinal hernia treatment (single or double)
- Prevent Hernia relapse

CAUTION/RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS: [1]

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.
- Adjust padding and strapping for a secure and comfortable fitting.

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