PRODUCT CARE

Keep away from sharp edges

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English

INTENDED USE :

Hot and Cold Pack is a convenient device to provide hot fomentation or cold compress.

Hot fomentation of the injured or inflamed area enhances the threshold of pain and thus reduces its perception. It has a synergistic effect with pain relieving drugs. Raising temperature of the injured tissue also enhances the blood supply and the healing process. Hot fomentation has a relaxing effect.

Cold compress helps in reduction of inflammation in injuries which protects by slowing the metabolic rate around the tissue, reduce oedema and bleeding. Cold compress helps in immediately lowering fever, in very high fever conditions. It can be used after an acute injury or surgical procedure.

CONSTRUCTION:

The gel pouch is made out of soft and flexible PU film, filled with a hydrophilic gel made out of edible polymers. This gel has high heat retention property.

APPLICATIONS: Hot fomentation

- Muscular and Joint Pain
- Backache
- Rheumatoid or other arthritis
- Geriatric care
- Sprains
- Sore, over worked muscles
- General comfort in extra cold conditions
- Abscess
- Boils

APPLICATIONS: Cold compress

- High fever
- Oedema
- Bleeding
- Inflammation and soft tissue injuries
- Regain working of an over worked muscle.
- Post surgical care
- Management of strains, sprains, bruises

INSTRUCTIONS:

- Use this product under guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergic rash or itching
 - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Keep out of reach of children.
- Rupture of gel beads doesn't affect the efficacy of the product.