

## PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.




Do not iron.



Do not bleach.

**Knee Caps**  
नी कैप  
Rodilleras  
Genouillères  
ضمادة الركبة

**REF D 04/23**

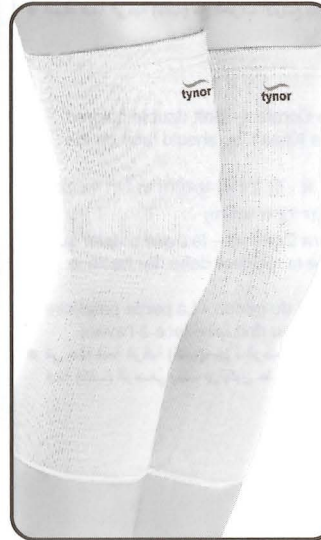

**Knee Cap**



**Knee Cap Comfeel**

**EC REP**

MT Promed Consulting GmbH  
Altenhofstrasse 80  
D-66386 St. Ingbert  
Germany  
tel.: +49 (0) 6894 581020  
fax: +49 (0) 6894 581021  
e-mail: info@mt-procons.com  
www.mt-procons.com

### Symbol Explanation :

- LOT** Batch code  
**M** Manufacturing date  
**M** Manufacturer  
**REF** Catalogue number

### Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India  
e-mail: marketing@tynorindia.com • www.tynorindia.com  
Customer Care : 0091-172-5288111

ISO 9001 | CRISIL | W.H.O. G.M.P. | ISO 13485 | CE | US FDA registered

UJM-D04/23-18-1

**How To Use**  
**Cómo Utilizar**



**कैसे उपयोग करें**  
**Comment utiliser:**  
**كيفية الاستخدام**

1.



- Hold the Knee Cap with the broader end upwards.
- चौड़े सिरे को ऊपर की ओर रखते हुए नी कैप को पकड़ें।
- Sujete la rodillera con el extremo más ancho hacia arriba.
- Tenez le genou avec l'extrémité la plus large vers le haut.

● أمسك قبة الركبة (كاب) مع الطرف الأوسع يكون صعوداً.

1a.



- In case of Knee Cap Comfeel - Soft double colored patellar portion of the Knee Cap should face on the front side.
- नी कैप कम्फील के मामले में - नी कैप का मुलायम दो रंगों का पट्टेला वाला हिस्सा सामने की ओर रहना चाहिए।
- En caso de la rodillera Comfeel - la parte patelar suave con un color doble de la rodillera debe dar hacia la parte delantera.
- En cas de comblement du genou - La partie patellaire molle et colorée du genou doit faire face à l'avant.
- في حالة قبة الركبة (كاب) - جزء الرضفي اللين المزدوج والملون من ضمادة قبة (كاب) الرضفي يجب أن يكون على الواجهة الأمامية.

**How To Use**  
**Cómo Utilizar**



**कैसे उपयोग करें**  
**Comment utiliser:**  
**كيفية الاستخدام**


2.



- Slide the foot into the product and pull up like socks.
- पैर को उत्पाद में डालें और फिर उत्पाद को मोजे की तरह चढ़ा लें।
- Deslice el pie en el producto y tire hacia arriba como si fueran unos calcetines.
- Glissez le pied dans le produit et remontez comme des chaussettes.

● أدخل القدم في المنتج واسحب إلى أعلى مثل لبس الجوارب

3.

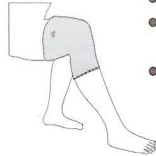

- Smoothen the wrinkles by massage action of the hand.
- हाथ फेरकर मिलवटों को ठीक कर लें।
- Elimine las arrugas mediante una acción masajeadora de la mano.
- Lisser les plis par l'action de massage de la main.

● قم بتنعيم التجاعيد عن طريق الدلك باليد

**English**

**INTENDED USE :**

Knee Caps are tubular supports widely used in orthopedic practice to provide mild compression, warmth & support to the knee joint, to allay pain and inflammation generally associated with old age, arthritis, injury, sports etc.

**Construction:**

Knee caps are made in double layer of Nylon fiber, Spandex and long staple Cotton yarn. Nylon improves life and suppleness of the product. Nylon is aesthetically more pleasing and offers color fastness. Cotton is dermophillic, and hypoallergenic. Interwoven, rubber thread and spandex provide the unique four way stretch to the product.

Knee Cap Comfeel is woven in next generation, computer numeric controlled, 3 dimensional looms, so that the product conforms to the complex anatomy of the knee with exactness. This offers a unique advantage of soft patella and fine grip to the edges and also enhances aesthetics of the product.

**APPLICATIONS:**

- Gives mild support to the weak knee.
- Gives warmth and compression to inflamed knees.
- Psychological support to geriatric patients.
- For arthritic knee conditions.
- Post-surgical rehabilitation.
- Sporting activities.

**RECOMMENDATION :**

- Use this product on recommendation of a doctor.

**INSTRUCTIONS:**

- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Consult your doctor in case of concurrent use of the device with excessive physical exertion, it can lead to restriction of performance of heart and lungs (risk of rise in blood pressure).
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.