PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing

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English

INTENDED USE :

Knee Cap with Patella Ring is a next generation knee support to provide stabilization, compression, grip and support to the weak or unstable knee.

CONSTRUCTION:

Woven on the state of art CNC looms to provide seam less, anatomically shaped tube, in a double layer.

Double layered fabric with nylon on the outside and cotton on inside offers unique advantages. Nylon improves life and suppleness of the product. Nylon is aesthetically more pleasing and offers color fastness. Inner cotton layer is dermatophilic and hypoallergenic, it quickly absorbs moisture and transports to the outer nylon layer where it quickly gets evaporated, keeps the skin cool and comfortable. Intermingled spandex provides the unique four way stretch to the fabric.

The silicone patellar ring eliminates the pressure on the patella, essential for conditions like chondromalacia or retro-patellar cartilage damage. Silicone insert massages gently, improves the blood circulation.

Flat spring multiaxial hinge enhances lateral support to the knee, prevents the edge roll over, and reduces buckling of fabric in the pit of the knee.

APPLICATIONS :

- Mild instability of the knee joint
- Warmth, compression and support to the weak knee.
- Swellings and effusions of the knee
- Post operative, post surgical or post trauma management
- Patello femoral syndrome
- Injury to meniscus
- Arthritic knee conditions
- Support to geriatric or very active sports knee.
- Patella chondroplasty

RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS: (1)

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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MT Promedt Consulting GmbH
**Tynor Orthotics Pvt. Ltd.**