## PRODUCT CARE

Hand wash with mild detergent and water below 30°C.

Do not wring the product

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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## Knee Support Sportif (Neoprene)

**Knee Support Sportif (Neoprene)** is a designer knee support to provide firm compression, mild support and warmth to the knee joint, to allay pain and inflammation, generally associated with old age, arthritis. Ideal for sports and for sports related injuries.

### CONSTRUCTION:

The body is made out of three layered Neoprene foam bonded fabric, with a four way stretch. The outer layers are made out of strong nylon fabric which ensures a long life, good aesthetics and color fastness. It is anatomically shaped with porous zone, which also ensures no bunching on knee bending.

The top and the bottom band gives controlled compression and a firm grip. Anterior patellar opening provides proper positioning of patella and ensures no compression of the patella.

### APPLICATIONS:

- For active sports people
- Sprains and strains
- For weak knees and geriatric patients
- For arthritic knee conditions
- Sports injury
- Post-surgical rehabilitation
- Patellar dislocations
- Patellar tendonitis
- Chondromalacia
- Resistant patella-femoral pain

▲ **CAUTION / RECOMMENDATION :** The product should be removed while sleeping.

### INSTRUCTIONS:

- Use this product on recommendation of a doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
  - Allergy, rash or itching
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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