

## PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not wring the product



Do not dry-clean.



Do not Iron.



Do not bleach.



Close hook loop fasteners before washing.

# tynor

Knee Wrap (Neoprene)

नी रैप (नेप्रिन)

Rodillera (De Neopreno)

Genouillère (Neoprene)

(نيوبرين)

REF

EC REP

MT Promed Consulting GmbH  
Altenhofstrasse 80  
D-66386 St. Ingbert  
Germany  
tel.: +49 (0) 6894 581020  
fax: +49 (0) 6894 581021  
e-mail: info@mt-procons.com  
www.mt-procons.com

### Symbol Explanation :



LOT Batch code



Manufacturing date



Manufacturer



Catalogue number

# tynor

## Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLP1, Mohali-160 055, (Pb) India  
e-mail: marketing@tynorindia.com • www.tynorindia.com  
Customer Care : 0091-172-5288111

ISO 9001 | CRISIL | W.H.O. G.M.P. | ISO 13485 | CE | US FDA registered

UM-J05-18-2

www.tynorindia.com

USER MANUAL

**How To Use**  
**Cómo Utilizar**



**कैसे उपयोग करें**  
**Comment utiliser:**  
**كيفية الاستخدام**

1.



- Place the product with wider side up and the patellar opening on the patella.
- उत्पाद के चौड़े हिस्से को ऊपर की तरफ रखें और पट्टेला वाले खुले हिस्से को पट्टेला पर रखें।
- Coloque el producto con su lado más ancho hacia arriba y el orificio de la tela en el área de la rótula sobre la rotula.
- Placez le produit avec le côté plus large vers le haut et l'ouverture patellaire sur la rotule.
- ضع الحزام مع جعل الجانب الأوسع لأعلى والفتحة الرضفية فوق الرضفة.

2.



- The three arms are encircled around the knee.
- तीनों भुजाएं घुटने के इर्द-गिर्द गोलाई में घुमाकर रखें।
- Las tres bandas se colocan en torno a la rodilla.
- Les trois bras sont entourés autour du genou.
- يتم تطويق الأذرع الثلاثة حول الركبة.

**How To Use**  
**Cómo Utilizar**



**कैसे उपयोग करें**  
**Comment utiliser:**  
**كيفية الاستخدام**

3.



- The hook tape is closed on to the product to a desired compression.
- हुक टेप को वांछित दबाव बनाकर बंद कर दें।
- Las hebillas se ajustan para obtener la compresión adecuada.
- La bande de crochet est fermée sur le produit à une compression désirée.
- تم بغلاق شريط الربط على الحزام للحصول على الضغط المطلوب.

**How To Use**  
**Cómo Utilizar**



**कैसे उपयोग करें**  
**Comment utiliser:**  
**كيفية الاستخدام**

3.



- The hook tape is closed on to the product to a desired compression.
- हुक टेप को वांछित दबाव बनाकर बंद कर दें।
- Las hebillas se ajustan para obtener la compresión adecuada.
- La bande de crochet est fermée sur le produit à une compresion désirée.
- قم بغلاق شريط الربط على الحزام للحصول على الضغط المطلوب.

**Knee Wrap (Neoprene)**

**English**

**INTENDED USE :**

**Knee Wrap (Neoprene)** is an anterior opening design which offers an advantage of easy application on swollen or asymmetric knees and to geriatrics. It allows customized compression, not possible in tubular products. Neoprene fabric provides firm compression, warmth & support to the knee to allay pain and inflammation, generally associated with old age, arthritis, sports injury etc.

**CONSTRUCTION:**

Made out of three layered Neoprene foam bonded fabric, with a four way stretch, top layer with unbroken loops offer the advantage of a very versatile closing system and a firm grip. Inner layer made from strong nylon fabric ensures a long life, good aesthetics and color fastness. Neoprene material provides good cushioning and four-way compression which ensure convenient flexion and twisting action to the knee area.

Its anterior opening design and convenient triple hook loop closing system ensure easy application, better grip and controlled compression around the knee.

Patellar opening and patellar padding provide proper support, positioning of patella and comfortable compression to the area.

**APPLICATIONS:**

- Sprain and strain of the knee
- Support to geriatric knees
- Arthritic knee conditions
- Sports injury
- Post-surgical rehabilitation
- Patellar dislocations
- Patellar tendonitis
- Chondromalacia
- Resistant patello-femoral syndrome



**CAUTION/RECOMMENDATION :**

- Use this product under strict guidance of a qualified doctor.
- The product should be removed while sleeping.

**INSTRUCTIONS:**

- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.