

PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.



Do not Iron.



Do not bleach.

EC REP

MT Proment Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mf-procons.com

Symbol Explanation :



Batch code



Manufacturing date



Manufacturer



Catalogue number


tynor

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JIPL, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO 9001 | CRISI | W.H.O. G.M.P. | ISO 13485 | CE | US FDA registered

www.tynorindia.com



Knee Cap Open Patella

नी कैप ओपन पैटेल्ला
Rodillera Con Rótula Abierta
Bouchon Du Genou Rotulien Ouvert
دعامة الركبة مفتوحة الرضفة



UM-D05-18-1


REF D 05

USER MANUAL

How To Use
Cómo Utilizar



कैसे उपयोग करें
Comment utiliser
كيفية الاستخدام

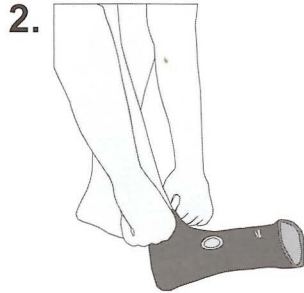

How To Use
Cómo Utilizar



कैसे उपयोग करें
Comment utiliser
كيفية الاستخدام



- Worn by a simple pull on application like socks, with the broader end facing upwards.
- मोजे की ही तरह इस एप्लिकेशन को बॉर्डर के छोर को ऊपर की ओर करके सामान्य रूप से खींच कर पहना जाता है।
- Se lleva puesta tirando simplemente de ellas similar a aplicaciones como calcetines, con el extremo más ancho dando hacia arriba.
- Porté par une simple application sur les chaussettes, l'extrémité la plus large étant orientée vers le haut.
- يمكن ارتداؤها بسهولة عن طريق سحبها على الساق مثل الجوارب، مع الطرف الأوسع يتجه لأعلى.

Knee Cap Open Patella

English

INTENDED USE :

Knee Cap open patella provides firm compression, warmth & support to the knee to allay pain and inflammation, generally associated with old age, arthritis, sports injury etc.

CONSTRUCTION:

Woven on the state of art circular looms, from high quality, nylon and spandex yarn, to get a four way stretching seamless tubes tapered from top to bottom. Strong Spandex thread runs spirally in the matrix of nylon locks to ensure its free movement to allow uniform compression even on uneven diameters of the limbs.

APPLICATIONS :

- Firm support and mild immobilization to the weak knee.
- Warmth for quick healing
- Compression to inflamed knees.
- Psychological support to geriatric patients.
- For arthritic knee conditions.
- Sports injury.
- Post-surgical rehabilitation.
- Patellar dislocations



RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.