PRODUCT CARE

Hand wash fabric with mild detergent and water below 30°C without wringing.

Wipe the plastic parts with moist lint free cloth and mild detergent

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

It is recommended to close hook loop closures before washing.

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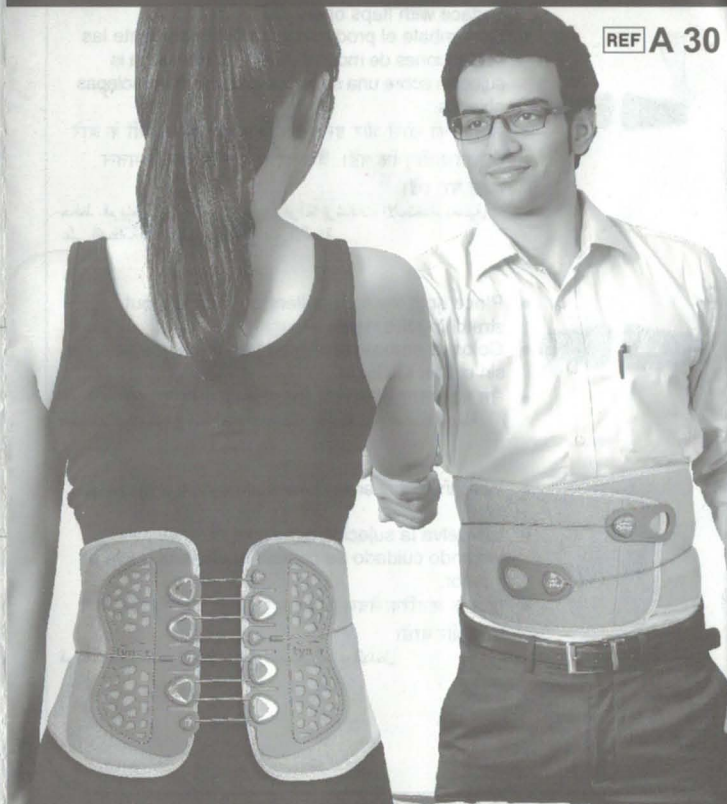
English

INTENDED USE:

Lacepull L.S. Belt is an advanced brace designed with semi rigid back splints and multiple pulley driven tightening mechanism to deliver controlled and independent compression to attenuate lower back pain. It ergonomically braces the waist, requires very less force, to achieve high compression and immobilization. Recommended specifically for postoperative stabilization and multi-level decompression.

CONSTRUCTION:

Made from aerated mesh fabric which is light in weight, highly porous and provides strong compression. Semi rigid splints provide perfect fit around the waist for effective decompression, immobilization and enhanced patient comfort.

APPLICATIONS

- Post-operative stabilization.
- Stable, non - displaced spinal fractures
- Spinal stenosis.
- Herniated discs
- Degenerative spinal pathologies
- Spondylololthesis.
- Spondyololysis
- Ligamentous disorders
- Low back muscle strain

INSTRUCTIONS: [1]

Use this product under strict guidance of a qualified doctor.

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - If pain increases or persists consult your doctor in case of concurrent use with another medical device.
- Consult your doctor in case of concurrent use with another medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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