PRODUCT CARE

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English

INTENDED USE :

Lumbo Lacepull Brace is an advanced brace designed with semi rigid posterior back panel and multiple pull driven pull mechanism to emphasize controlled and independent pressuring effects, to stabilize lower back pain. It ergonomically braces waist, requires lesser force for old aged users to achieve high compression and immobilization. Recommended specifically for postoperative stabilization and multi-level decompression.

CONSTRUCTION:

Made from aerated mesh fabric which is, light in weight, highly porous and has a strong compression. Anatomical back panel provides perfect fit around the waist for effective decompression, immobilization and enhanced patient comfort.

APPLICATIONS:

- Post-operative stabilization
- Stable, non - displaced spinal fractures
- Spinal stenosis
- Herniated discs
- Degenerative spinal pathologies
- Spondylololthesis
- Spondylololysis
- Ligamentous disorders
- Low back muscle strain

RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Product be preferably worn over a cotton lining or an under garment.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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