PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

Batch code

Manufacturing date

Manufacturer

Catalogue number

tynor

Manufactured by:

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, JPL Ind. Area, Mohali - 160 055 (Pb) India
Consumer Care Cell: 1800-1212-656
E-mail: care@tynor.in or write to manufacturer
www.tynorindia.com

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English

INTENDED USE :

Lumbo Sacral Belt is designed with flexible back splints, in order to provide comfortable support to the back and to allay low back pain. The belt can be worn during daily routines and as a prophylactic aid. Excellent aesthetics ensures product can be worn in any social setting.

CONSTRUCTION:

Made from extra strong highly porous orthopedic webbing containing heat resistant rubber with high modulus of elasticity. Hook loop closures provide excellent fitting of the belt around the waist and its quick easy application and removal. Alligator tabs at the edge of hook loop closures are aesthetically appealing and give a nice holding grip. Anti skid holder strap is provided for easy closure of the belt. Tough & flexi Nylon bars and polypropylene strip at the back provide sufficient firmness to ensure good immobilization of the lumbo sacral region. Flexi splints take the shape of the spine on wearing and support the back in that position.

APPLICATIONS:

- Minor spondylolysis, Lumbar spondylitis.
- Early Stage osteochondrosis, spondyloarthrosis
- Inter vertebral disc syndrome
- Chronic low back pain, postural fatigue, postural deformities and mechanical stresses due to poor posture
- Osteoporetic pain of the lumbar spine
- Comfort during long working hours and travel.
- Placebo effect keeps the patient spontaneously follow the Do's and Don'ts of the doctor
- Post operative rehabilitation after disc surgery, lumbar fusion and fracture fixation
- Stabilization during Vertebral Fracture and injuries
- Temporary immobilization for accident victims during extrication and transportation.

CAUTION/RECOMMENDATION:

- Use this product on recommendation of a doctor.
- Base of the Belt should coincide with sacrum at the level of S2 vertebrae. Center of the back panel be located on the center of the vertebral column.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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