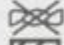
PRODUCT CARE

-  Hand wash with mild detergent and water below 30°C.
-  Do not wring the product
-  Dry in shade on a flat surface.
-  Do not dry-clean.
-  Do not iron.
-  Do not bleach.
-  Close hook loop fasteners before washing.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

-  LOT Batch code
-  Manufacturing date
-  Manufacturer
-  REF Catalogue number

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
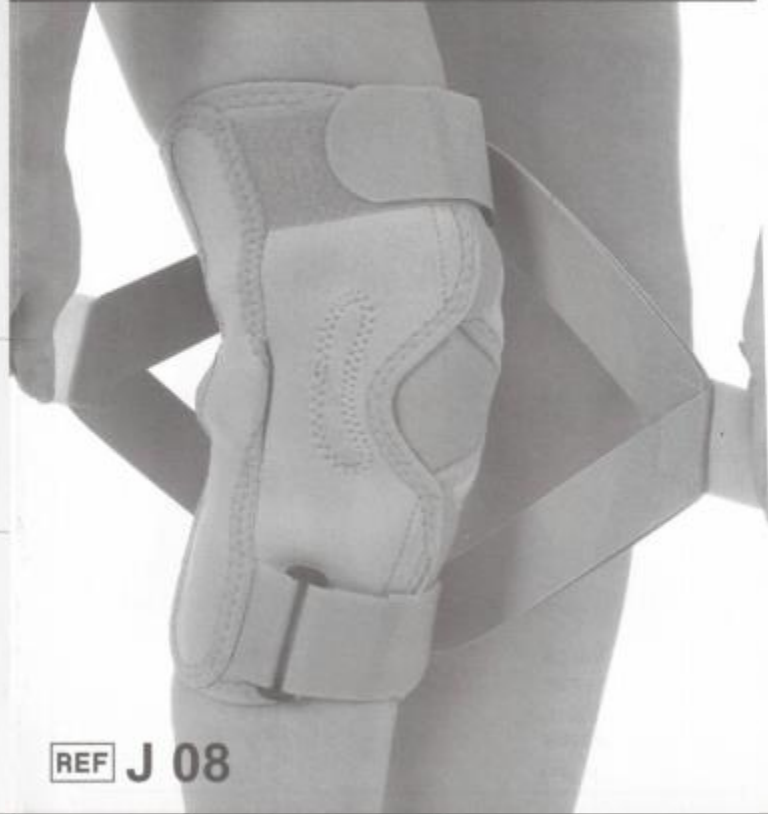
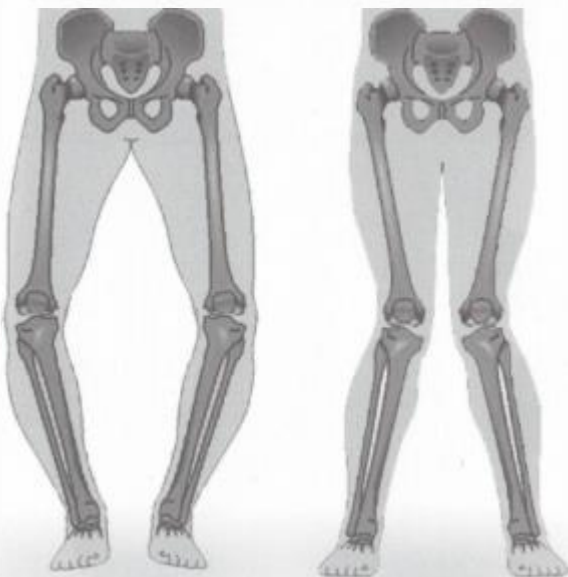
Valgus
वल्वस
Valgo
Valgus
أزوح

Correct orientation & position of OA Knee Support

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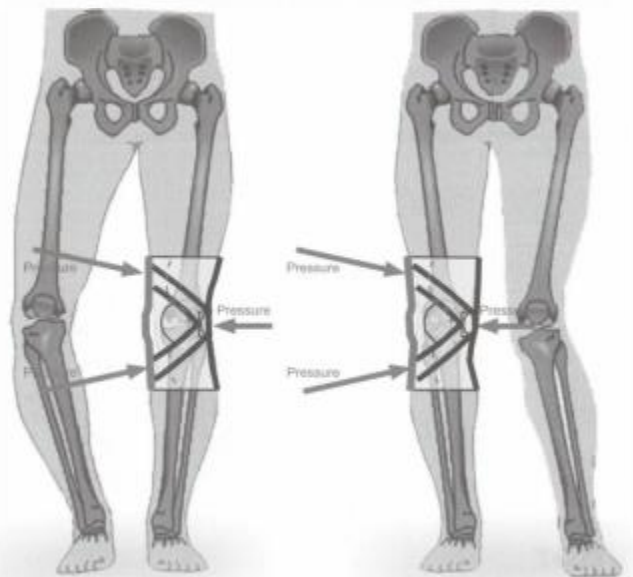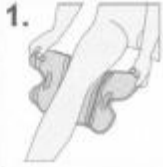
O.A. Knee Support (Neoprene)

English

O.A. Knee Support (Neoprene) is designed scientifically to provide counter force to correct varus or valgus thrust along with some offloading action. It helps to allay pain within a few weeks of use. It can also correct mild to medium varus/ valgus deformity of the knee with a prolonged use. It offers the advantage of controlled compression around the knee and the rigid side support of a splint. It allows normal flexion and free movement of the knee joint.

CONSTRUCTION:

The body is made out of three layered Neoprene foam bonded fabric, with a four way stretch, the outer layers are made out of strong nylon fabric which ensures a long life, good aesthetics and color fastness. Anterior opening design allows easy application, particularly in geriatric patients. Hook loop closures provide controlled compression and close contoured fitting apart from easy application and removal. Strong hinged splints provide lateral stability and support, but allow normal flexion of the knee joint.

APPLICATIONS:

- Mild to moderate osteoarthritis of the knee
- Varus /Valgus thrust
- General pain relief to the knee
- Chronic laxity or instability
- Support to weak or geriatric knee
- Sports injury
- Chondromalacia
- Patella-femoral pain
- Post-operative care after arthroscopic surgery
- Knee conditions requiring load reduction

CAUTION/RECOMMENDATION :

- The product should be removed while sleeping.
- Direction of the counterforce should be understood carefully before application.

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case pain increases.
- Discontinue use in case of, allergy rash or itching.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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