## PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Do not wring the product

Dry in shade on a flat surface.

Do not bleach.

EC REP

MT Promedt Consulting GmbH  
Altenhofstrasse 80  
D-66386 St. Ingbert  
Germany  
tel.: +49 (0) 6894 581020  
fax: +49 (0) 6894 581021  
e-mail: info@mt-procons.com  
www.mt-procons.com

### Symbol Explanation :

- LOT Batch code  
Manufacturing date  
Manufacturer  
REF Catalogue number

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**English**

Decide accurately the width and type of orthostokinet, "Absorbent or Non Absorbent". Measure the area of application, and cut the required length accordingly. Roll it over to form a ring. Unroll on the desired area. Apply the POP cast, Orthosis etc. as the case may be.

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## Ortho Stockinete

### English

Made from the finest long staple cotton yarn, woven on state of art circular looms. It is available as non absorbent greige, absorbent beige & ultra white. The absorbent varieties are treated for hypoallergenicity during the process of dyeing washing and finishing. Knitting in loose loops provides an inbuilt property of four way stretch to the fabric.

#### APPLICATIONS:

- As an underliner for POP cast
- As a hypo allergenic underliner for OrthoticAids
- As a construction material for stump liners
- In burns treatment
- In specialized bandaging
- Protection for photo sensitive skins
- As a generic material for creating fracture supports and binders

**CAUTION/RECOMMENDATION :** Decide the size and the type i.e. Absorbent or Non absorbent carefully before application.

#### INSTRUCTIONS:

- Use this product on recommendation of a doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.
- Product be preferably worn over a cotton lining or an under garment.

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